

ALL INFORMATION IN
THIS DRAWING SET IS
PRELIMINARY, SUBJECT
TO CHANGE!

[illegible]

DISCLAIMER

GENERAL SPECIFICATIONS

- GEHC is not responsible for the installation of developers and associated equipment, lighting, cassette trays and protective screens or derivatives not mentioned in the order.
- The final study contains recommendations for the location of GEHC equipment and associated devices, electrical wiring and room arrangements. When preparing the study, every effort has been made to consider every aspect of the actual equipment expected to be installed.
- The layout of the equipment offered by GEHC, the dimensions given for the premises, the details provided for the pre-installation work and electrical power supply are given according to the information noted during on-site study and the wishes expressed by the customer.
- The room dimensions used to create the equipment layout may originate from a previous layout and may not be accurate as they may not have been verified on site. GEHC cannot take any responsibility for errors due to lack of information.
- Dimensions apply to finished surfaces of the room.
- Actual configuration may differ from options presented in some typical views or tables.
- If this set of final drawings has been approved by the customer, any subsequent modification of the site must be subject to further investigation by GEHC about the feasibility of installing the equipment. Any reservations must be noted.
- The equipment layout indicates the placement and interconnection of the indicated equipment components. There may be local requirements that could impact the placement of these components. It remains the customer's responsibility to ensure that the site and final equipment placement complies with all applicable local requirements.
- All work required to install GEHC equipment must be carried out in compliance with the building regulations and the safety standards of legal force in the country concerned.
- These drawings are not to be used for actual construction purposes. The company cannot take responsibility for any damage resulting therefrom.

CUSTOMER RESPONSIBILITIES

- It is the responsibility of the customer to prepare the site in accordance with the specifications stated in the final study. A detailed site readiness checklist is provided by GEHC. It is the responsibility of the customer to ensure all requirements are fulfilled and that the site conforms to all specifications defined in the checklist and final study. The GEHC Project Manager of Installation (PMI) will work in cooperation with the customer to follow up and ensure that actions in the checklist are complete, and if necessary, will aid in the rescheduling of the delivery and installation date.
- Prior to installation, a structural engineer of record must ensure that the floor and ceiling is designed in such a way that the loads of the installed system can be securely borne and transferred. The layout of additional structural elements, dimensioning and the selection of appropriate installation methods are the sole responsibility of the structural engineer. Execution of load bearing structures supporting equipment on the ceiling, floor or walls are the customer's responsibility.

THE UNDERSIGNED, HEREBY CERTIFIES THAT I HAVE READ AND APPROVED THE PLANS IN THIS DOCUMENT.		
DATE	NAME	SIGNATURE

CUSTOMER SITE READINESS REQUIREMENTS

REQUIRED MANUALS FOR SYSTEM PRE-INSTALLATION	
Description	Document Number*
Product specific Pre-installation Manual	Refer to cover page
Magnet Room Venting	5850263
RF Shielded Room Pre-installation Requirements for MR systems	5850260
IEC Electromagnetic Compatibility	5850261
Acoustic Room Details	5850262
Magnet Venting Conformance Assessment Form	2705036
*documents can be accessed in multiple languages at https://www.gehealthcare.com/support/manuals	

- A mandatory component of this drawing set is the GE HealthCare Pre-installation manual. Failure to reference the Pre-installation manual will result in incomplete documentation required for site design and preparation.
- The items on the GE HealthCare Site Readiness Checklist **DOC2949060** and Worksheet **DOC2949068** are REQUIRED to facilitate equipment delivery to the site. Equipment will not be delivered if these requirements are not satisfied.
 - Any deviation from these drawings must be communicated in writing to and reviewed by your local GE HealthCare installation project manager prior to making changes.
 - Make arrangements for any rigging, special handling, or facility modifications that must be made to deliver the equipment to the installation site. If desired, your local GE HealthCare installation project manager can supply a reference list of rigging contractors.
 - New construction requires the following;
 1. Secure area for equipment,
 2. Power for drills and other test equipment,
 3. Restrooms.
 - Provide for refuse removal and disposal (e.g. crates, cartons, packing)
 - It is required to minimize vibrations within the scan room. It is the customer's responsibility to contract a vibration consultant/engineer to implement site design modifications to meet the GE vibration specification. Refer to the system Pre-installation manual for vibration specifications.

MRI SITE PLANNING REMINDERS

Please refer to pre-installation checklist in pre-installation manual listed on the cover sheet for items critical to image quality.

- 1. The layout should be arranged so that the 9g line is contained to the magnet room. If not possible, a barrier is recommended to prevent entry to the area.
- 2. The spaces around, above, and below the magnet must be reviewed for effects of the 9g, 5g, 3g, 1g, and .5g fields. Refer to the proximity limit chart in the MR pre-installation manual referenced on the cover sheet.
- 3. For moving metal, the restriction lines typically extend outside of the MR space. Please confirm there are no moving metal concerns within these areas.
- 4. For vibration, analysis to be completed as required per pre-installation manual.
- 5. For EMI, review the site for the location of the main electrical feeders, AC devices, or distribution systems. An EMI study is recommended if large AC systems are nearby.
- 6. Details of the floor below the magnet must be reviewed. The structural engineer must verify that the quantity of steel in the volume 10ft [3.1m] x 10ft [3.1m] x 13in [.3m] deep (below the magnet) does not exceed the allowable steel content as given in the MR pre-installation manual referenced on the cover sheet.
- 7. Remove, cover, or fill-in abandoned ducts or troughs from the Equipment and Magnet rooms. Access/computer room flooring in the Equipment room can either be removed or assessed and reinforced to support heavier cabinets.

Responsibility for the coordination, design, engineering, and site preparation resides with the customer and their project architects and contractors. GEHC does not, by providing reviews and furnishing comments and assistance, accept any responsibility beyond its obligations as defined in the MR system, sale/purchase agreement.

IMAGE QUALITY CONSIDERATIONS

Broadband RF noise is a single transient or continuous series of transient disturbances caused by an electrical discharge. Low humidity environmental conditions will have higher probability of electrical discharge. The electrical discharge can occur due to electrical arcing (micro arcing) or merely static discharge. Some potential sources capable of producing electrical discharge include:


- Loose hardware/fasteners vibration or movement (electrical continuity must always be maintained)
- Flooring material including raised access flooring (panels & support hardware) and carpeting
- Electrical fixtures (i.e. Lighting fixtures, track lighting, emergency lighting, battery chargers, outlets)
- Ducting for HVAC and cable routing
- RF shield seals (walls, doors, windows etc.)

For additional information regarding image quality, refer to the pre-installation manual listed on the cover sheet.

MAGNETIC INTERFERENCE SPECIFICATIONS


- The customer must establish protocols to prevent persons with cardiac pacemakers, neurostimulators, and biostimulation devices from entering magnetic fields of greater than 5 gauss (exclusion zone).
- Main power transformers must remain outside the 3 gauss field. EMI < 40mG RMS AC at the magnet location. EMI <9.39mG DC.
- Potential exists under fault conditions that the 5 gauss line may expand radially to 11.48 ft. [3.5 m] and axially to 14.76 ft. [4.5 m] for 1 second or less. It should be noted that normal rampdowns or magnet rundown unit initiated quenches will not cause the magnetic field to expand.
- It is recommended every site consider the event of a quench and plan accordingly (such as placing 5 gauss warning signs at expanded locations).
- The ferrous metal objects listed below must not move into or inside of the moving metal sensitivity line during scans.

TYPICAL MOVING MAGNETIC MASS	DISTANCE RADIALLY		DISTANCE AXIALLY	
Carts, Gurneys 100-400 lbs [45-182 kg]	3 Gauss line		3 Gauss line	
Forklifts, small elevator, cars, minivans vans, pickup trucks, ambulances (objects greater than 400 lbs [182 kg])	15.5 FT	4.72 M	21.0 FT	6.4 M
Buses and trucks (dump, tractor trailer, utility, fire trucks)	18.1 FT	5.52 M	24.5 FT	7.47 M

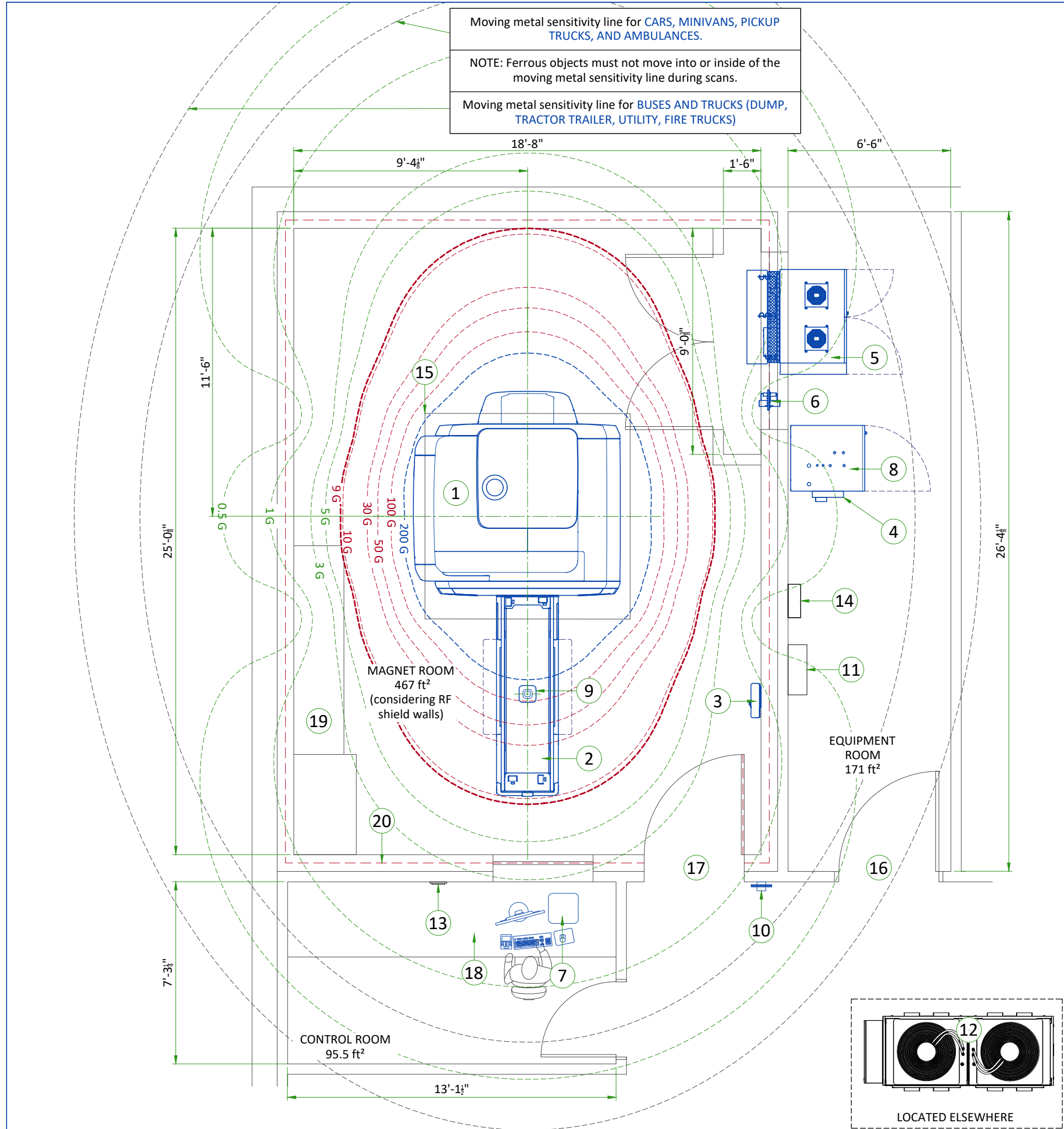


LEGEND						
A	GE SUPPLIED			D	AVAILABLE FROM GE	
B	GE SUPPLIED/CONTRACTOR INSTALLED			E	EQUIPMENT EXISTING IN ROOM	
C	CUSTOMER/CONTRACTOR SUPPLIED AND INSTALLED			*	ITEM TO BE REINSTALLED FROM ANOTHER SITE	
200 GAUSS				9 GAUSS		
100, 50, 30, 10 GAUSS				5, 3, 1, 0.5 GAUSS		
BY	ITEM	DESCRIPTION	MAX HEAT OUTPUT (BTU/h)	WEIGHT (lbs)	MAX HEAT OUTPUT (W)	WEIGHT (kg)
A	1	MAGNET (PM)	9865	10417	2891	4725
A	2	PATIENT TABLE (PT)	-	717	-	325
A	3	MAGNET RUNDOWN UNIT (MRU)	-	11	-	5
A	4	MAGNET MONITOR (MON)	205	8	60	3.6
A	5	INTEGRATED SYSTEMS CABINET (ISC)	27125	2762	7950	1253
A	6	PENETRATION PANEL (PP)	-	-	-	-
A	7	OPERATOR CONSOLE (CAB)	205	-	60	-
A	8	INTEGRATED COOLING CABINET (ICC)	6824	1257	2000	570
A	9	IN-ROOM CAMERA (IRC)	-	-	-	-
A	10	METAL DETECTOR (HAND HELD)	-	-	-	-
D	11	WATER FILTER	-	-	-	-
D	12	DIMPLEX CHILLER	153540	5367	45000	2434
D	13	REMOTE GRAPHIC DISPLAY (RGD)	-	-	-	-
C	14	FACILITY DISCONNECT (FD)	-	-	-	-
C	15	MAGNET ACCESS 2.5m x 2.5m [98.5in x 98.5in]	-	-	-	-
C	16	MINIMUM OPENING FOR EQUIPMENT DELIVERY IS 1016 mm x 2083 mm [40 in x 82 in], CONTINGENT ON A 1829 mm [72 in] CORRIDOR WIDTH				
C	17	MINIMUM OPENING FOR EQUIPMENT DELIVERY IS 1092 mm x 2083 mm [43 in x 82 in] CONTINGENT ON A 2438 mm [96 in] CORRIDOR WIDTH				
C	18	COUNTER TOP FOR EQUIPMENT- PROVIDE GROMMETED OPENINGS AS REQUIRED TO ROUTE CABLES				
C	19	BASE CABINET FOR STORAGE OF: SURFACE COILS, PATIENT POSITIONING PADS, PHANTOMS, ETC.				
C	20	DEFINE RF SHIELD'S INSET ACCORDING TO PROVISIONS MADE BY THE RF SHIELD VENDOR				
		RF SHIELD - 100 dB ATTENUATION				
EXAM ROOM HEIGHT						
FINISHED FLOOR TO SLAB HEIGHT					TBD	
FALSE CEILING HEIGHT					8'-9"	

SIDE VIEW WITH MAGNETIC FIELD

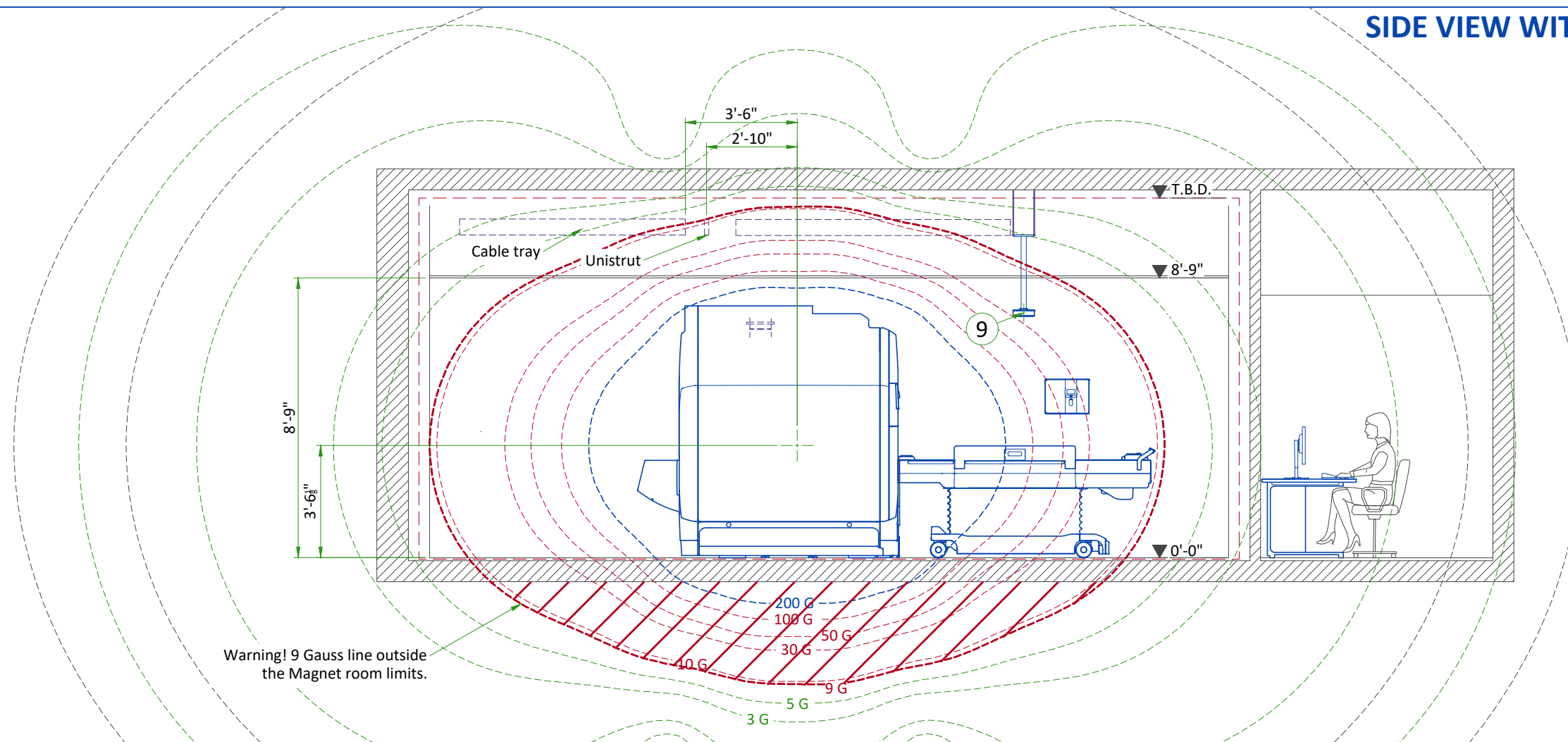

FRONT VIEW WITH MAGNETIC FIELD

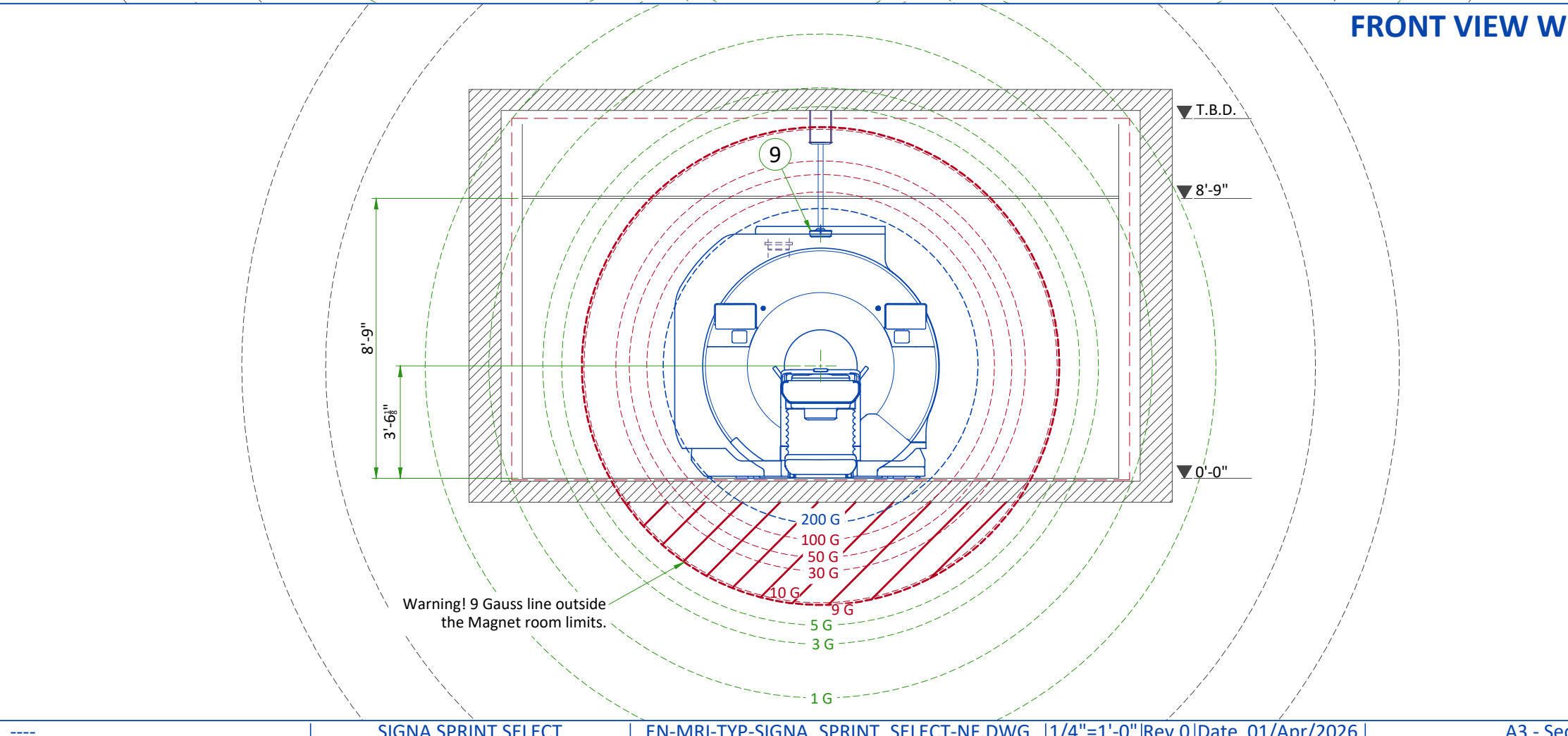


ACOUSTICS SPECIFICATIONS

Acoustic and vibroacoustic information is provided for site planning and architectural design activities. It is the customer's responsibility to hire a qualified acoustic engineer for solutions to further attenuate this transmitted noise and vibration, if required. The actual room noise level may vary based on room design, optional equipment, and usage:

Control Room: 62dBA
Equipment Room: 80dBA
Magnet Room: 122dBA*
(maximum sound pressure level at magnet bore isocenter)

* Frequency: 20 Hz to 20kHz


FREQUENCY (Hz)	AMPLITUDE (m/s²)
2	0.05
10	0.05
20	0.35
30	0.35
35	0.2
50	0.2

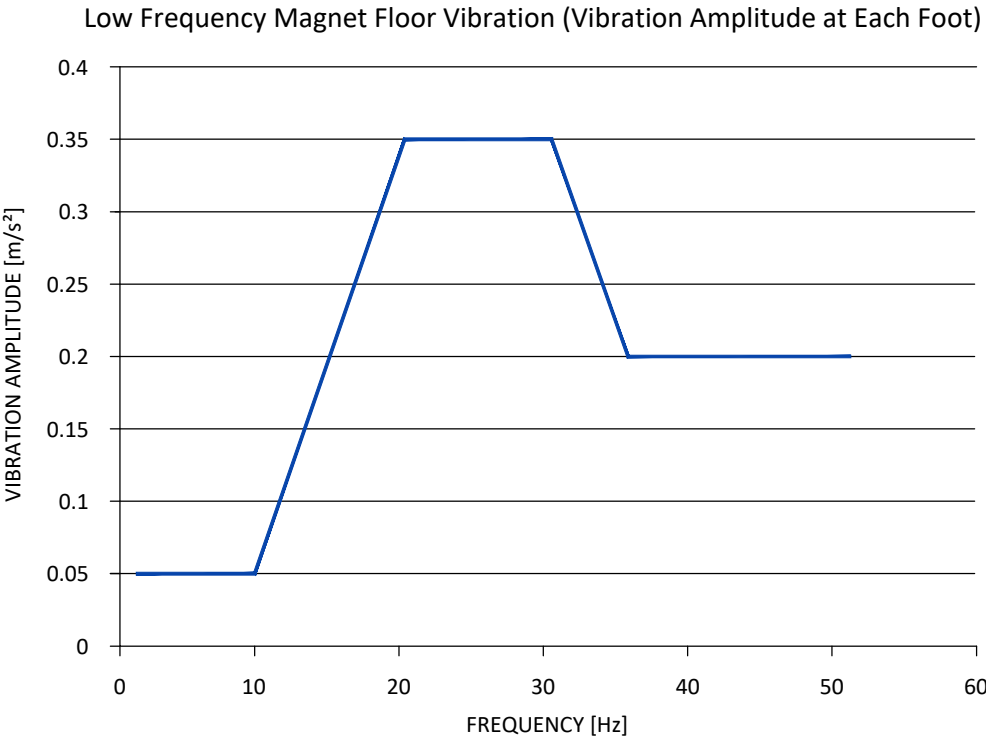

ISOGAUSS PLOTS

* The isogauss contour plots depicted on this drawing represent magnetic fringe fields resulting from the normal operation of the magnet provided with the MR system. The actual magnetic field intensity at any point in the vicinity of the magnet when installed may vary from the contour plots due to factors such as the concentrating effects of nearby ferrous objects ambient magnetic fields, including the earth's magnetic field. Therefore, the contours shown are only approximations of actual field intensities found at a corresponding distance from the magnet's isocenter.

SOUND PRESSURE SPECTRAL DISTRIBUTION

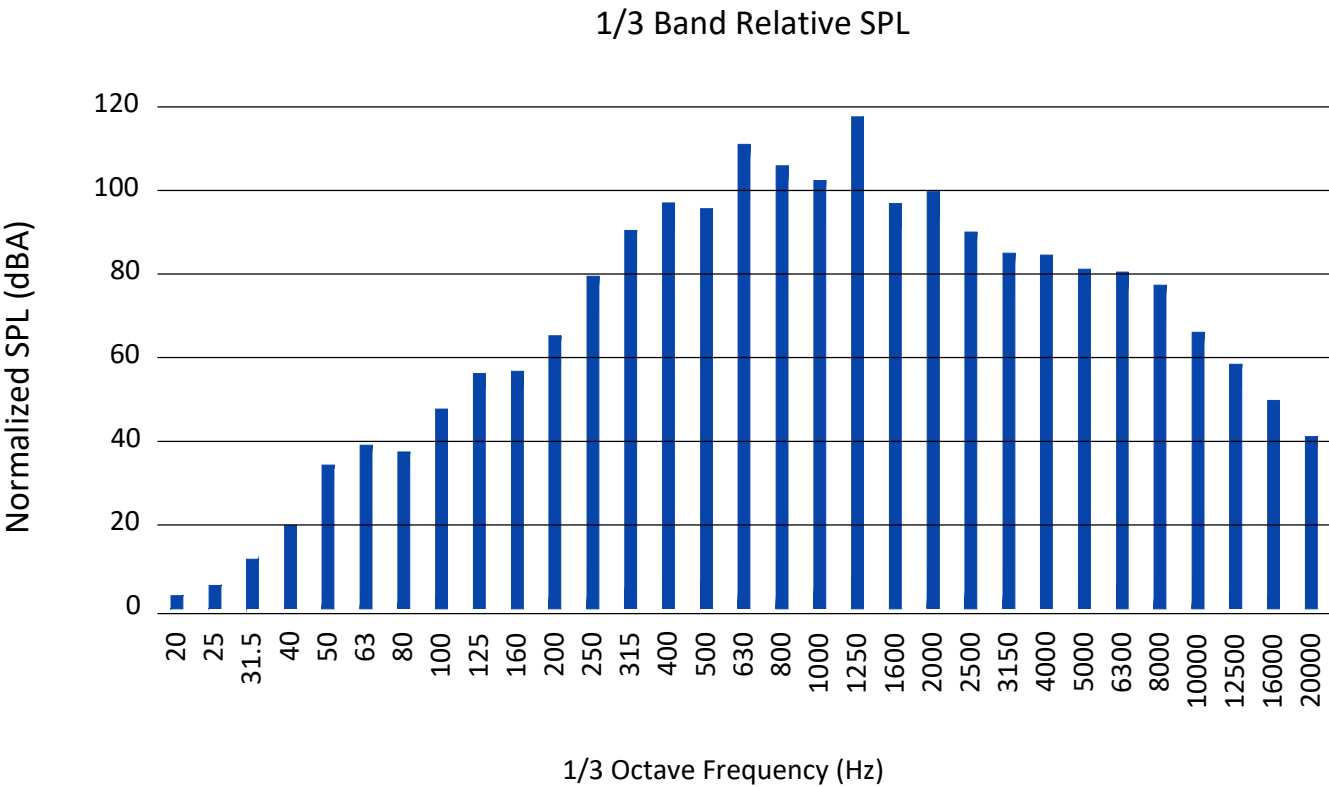

MAGNETIC PROXIMITY LIMITS

Gauss (mT) Limit	Equipment
0.5 gauss (0.05mT)	Nuclear camera
1 gauss (0.1mT)	Positron Emission Tomography scanner, Linear Accelerator, Cyclotrons, Accurate measuring scale, Analog image intensifiers, Bone Densitometers, Video display (tube), CT scanner, Ultrasound, Lithotripter, Electron microscope
3 gauss (0.3mT)	Power transformers, Main electrical distribution transformers
5 gauss (0.5mT)	Cardiac pacemakers, Neurostimulators, Biostimulation devices
10 gauss (1mT)	Magnetic computer media, Line printers, VCRs, Film processor, X-ray tubes, Emergency generators, Commercial laundry equipment, Food preparation area, Water cooling equipment, HVAC equipment, Major mechanical equipment room, Credit cards, watches, and clocks, Air conditioning equipment, Fuel storage tanks, Motors greater than 5 horsepower
50 gauss (5mT)	Metal detector for screening, LCD panels, Telephones
No Limit	Digital Detectors

The customer must provide detail defining ferrous material below the magnet to the Project Manager so the GE Healthcare MR Siting and Shielding team can review for compliance.

STEEL MASS LIMITS TO MAGNET ISOCENTER (3x3 m [10x10 ft] AREA UNDER MAGNET)			
Limits Of Steel Mass		Distance Below Top Surface Of Floor	
kg/m²	lbs/ft²	mm	in
0	0	0 - 76	0-3
9.8	2	76 - 127	3-5
14.7	3	127 - 254	5-10
39.2	8	254 - 330	10-13
98.0	20	330+	13+


The actual field strength can be affected by Magnetic shielding, Earth's magnetic field, other magnetic fields and stationary or moving metal. This information must be used to evaluate potential site interaction of GE Healthcare equipment with other non-GE Healthcare equipment. Magnetic shielding can be installed to prevent interaction between the magnet and nearby sensitive devices. The GE Healthcare Project Manager of Installation (PMI) can work with the customer to coordinate the magnetic shielding site evaluation. The customer is responsible for installation of all magnetic shielding.

PENETRATION PANEL WALL OPENINGS


View from equipment room side.

Refer to Equipment Layout page for site specific orientation of Penetration Panel.
Integrated System Cabinet must be covered on both sides for safety.


OPTION 2: PENETRATION PANEL ABOVE ISC

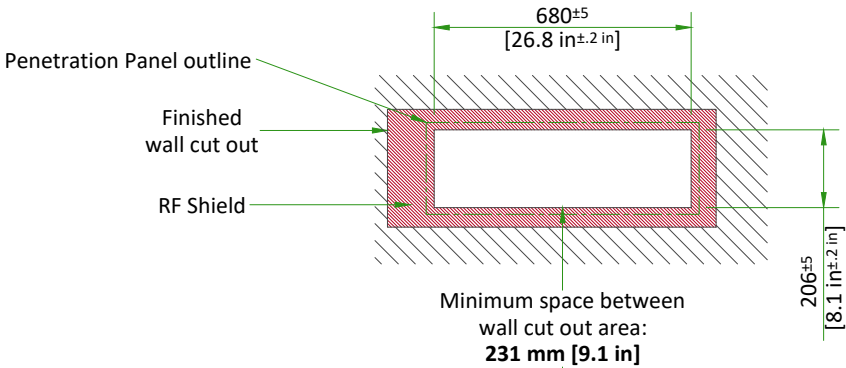

OPTION 1: PENETRATION PANEL, EITHER SIDE OF ISC

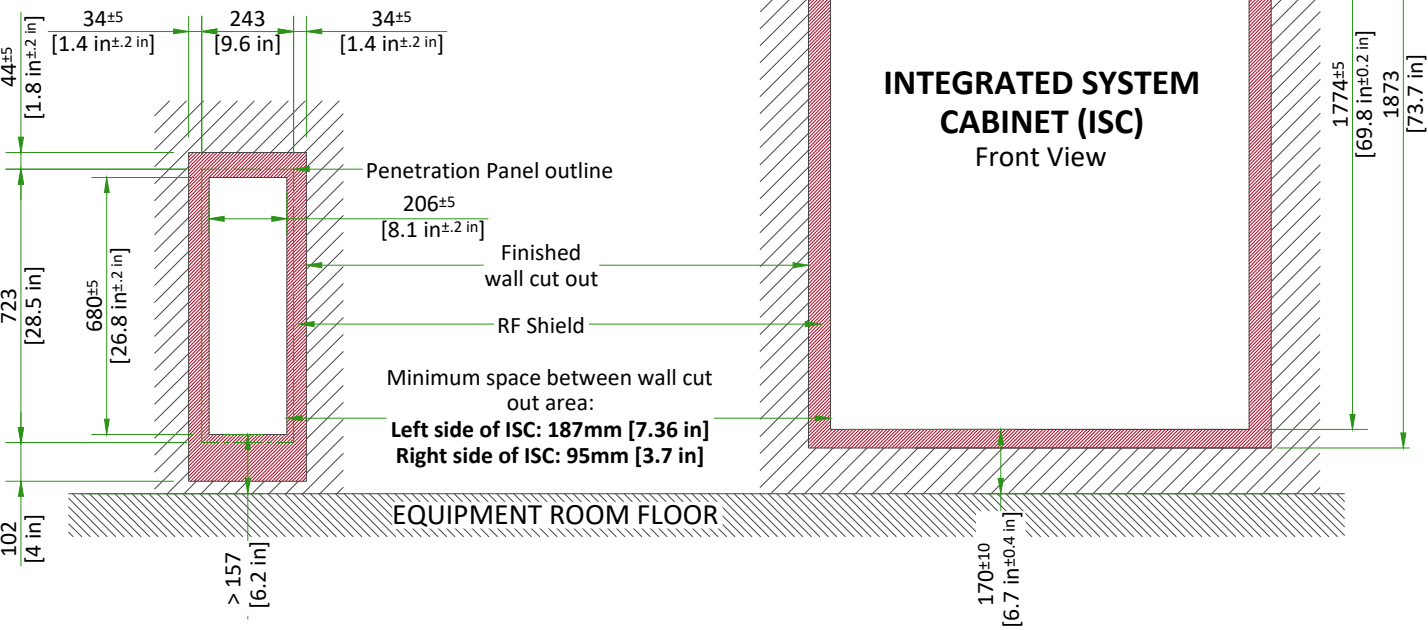


PENETRATION PANEL CLOSET

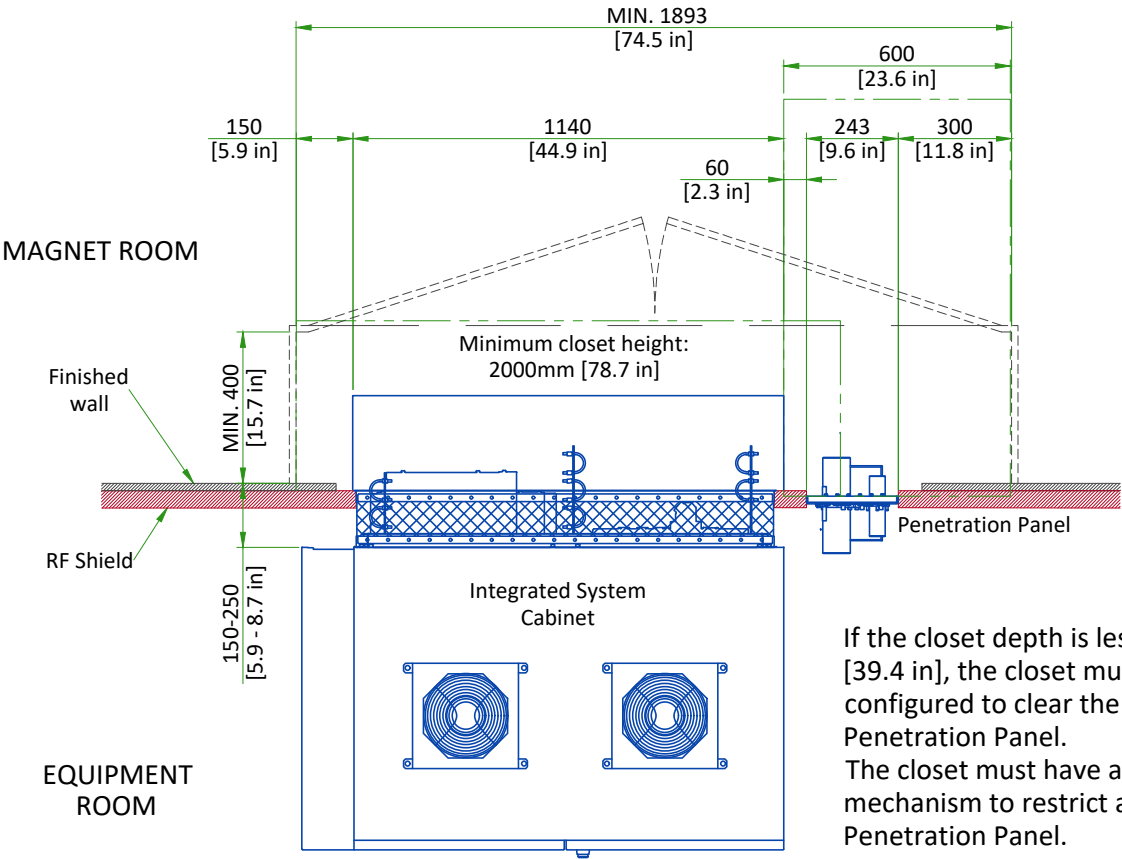


If the closet depth is less than 1000mm [39.4 in], the closet must have door(s) configured to clear the service area for the Penetration Panel.
The closet must have a mechanical locking mechanism to restrict access to the Penetration Panel.

MAGNET ENCLOSURE

FRONT VIEW

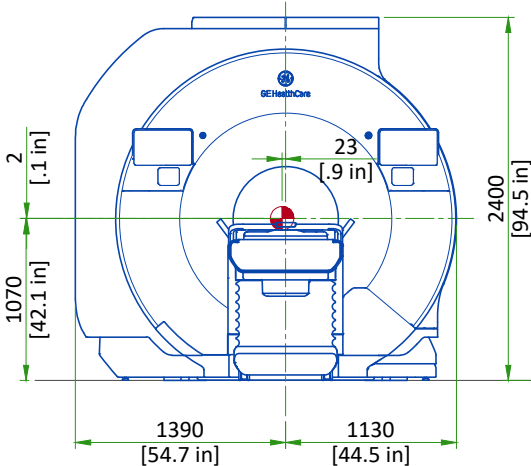


SIDE VIEW

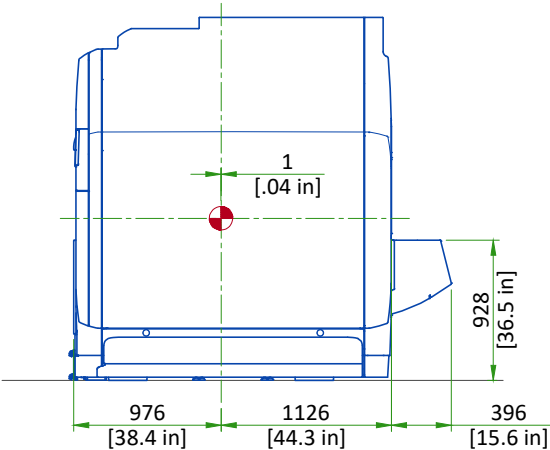


Center of gravity
SCALE 1:50

PATIENT TABLE


TOP VIEW

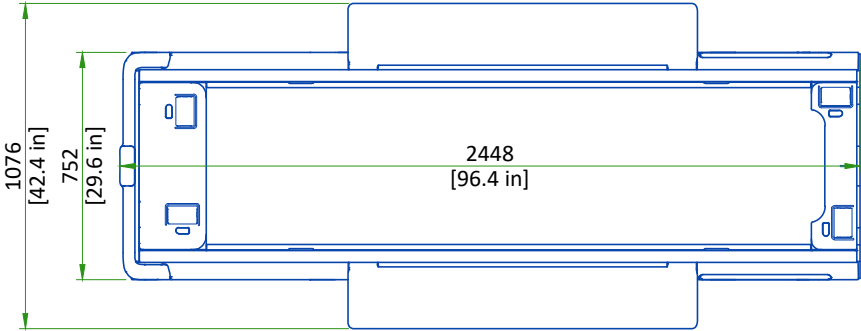

SIDE VIEW

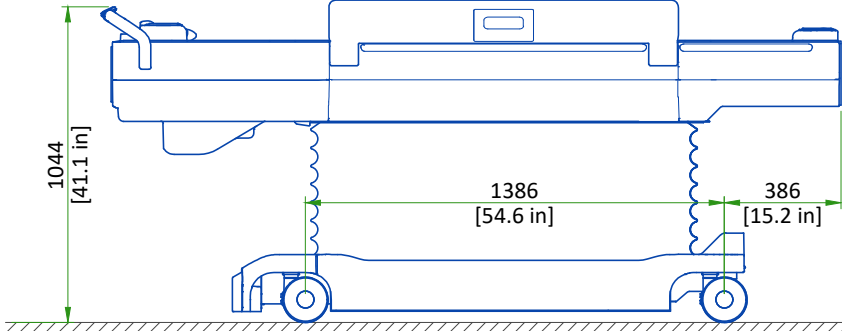


FRONT VIEW

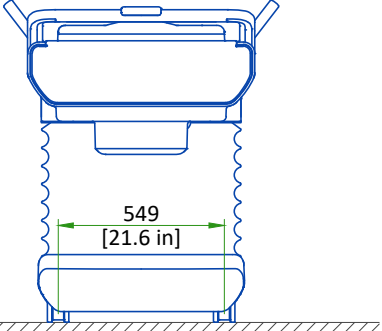

Patient is not on the table. Table is at the highest position. Refer to Preinstallation Manual for more information.
SCALE 1:25


INTEGRATED SYSTEM CABINET

SECTION VIEW

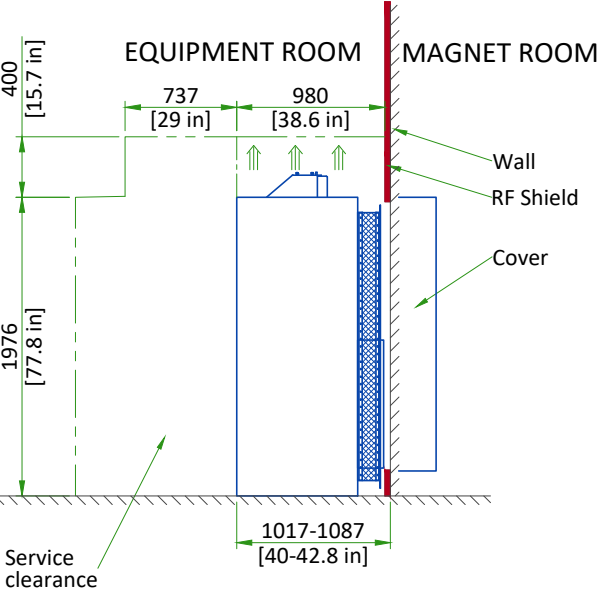


Air flow
Center of gravity


PLAN VIEW

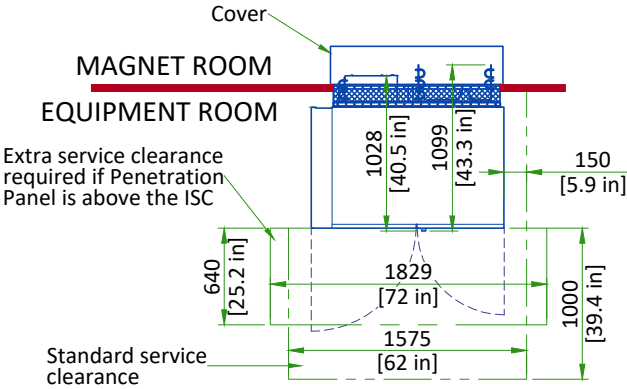

SIDE VIEW

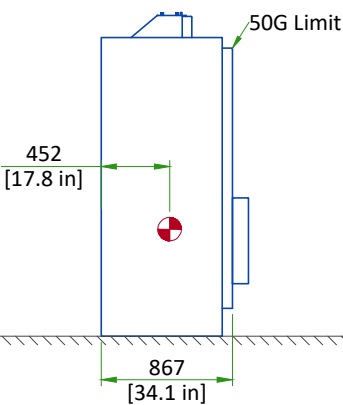


FRONT VIEW

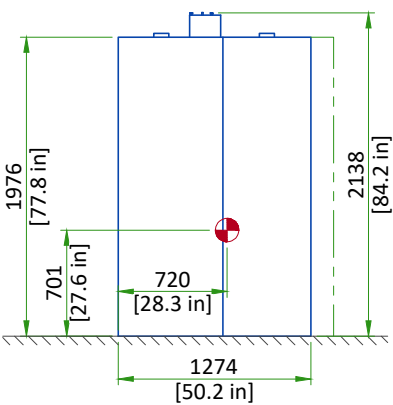


PENETRATION PANEL

SIDE VIEW

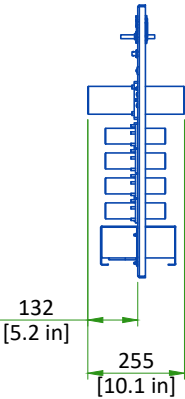


FRONT VIEW

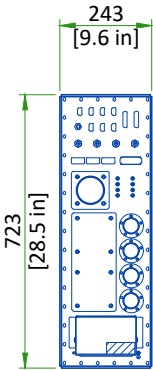


TOP VIEW

Minimum Service Clearance at each side is 60 mm [2.4 in]. The Penetration Panel (PP) does not need to be centered in the Service Clearance.

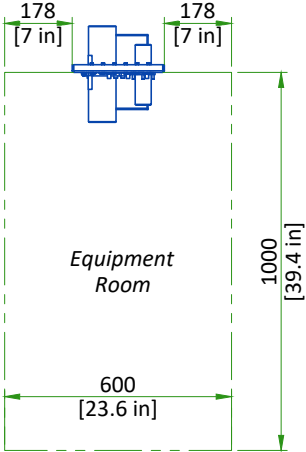

SCALE 1:20


INTEGRATED COOLING CABINET (ICC)

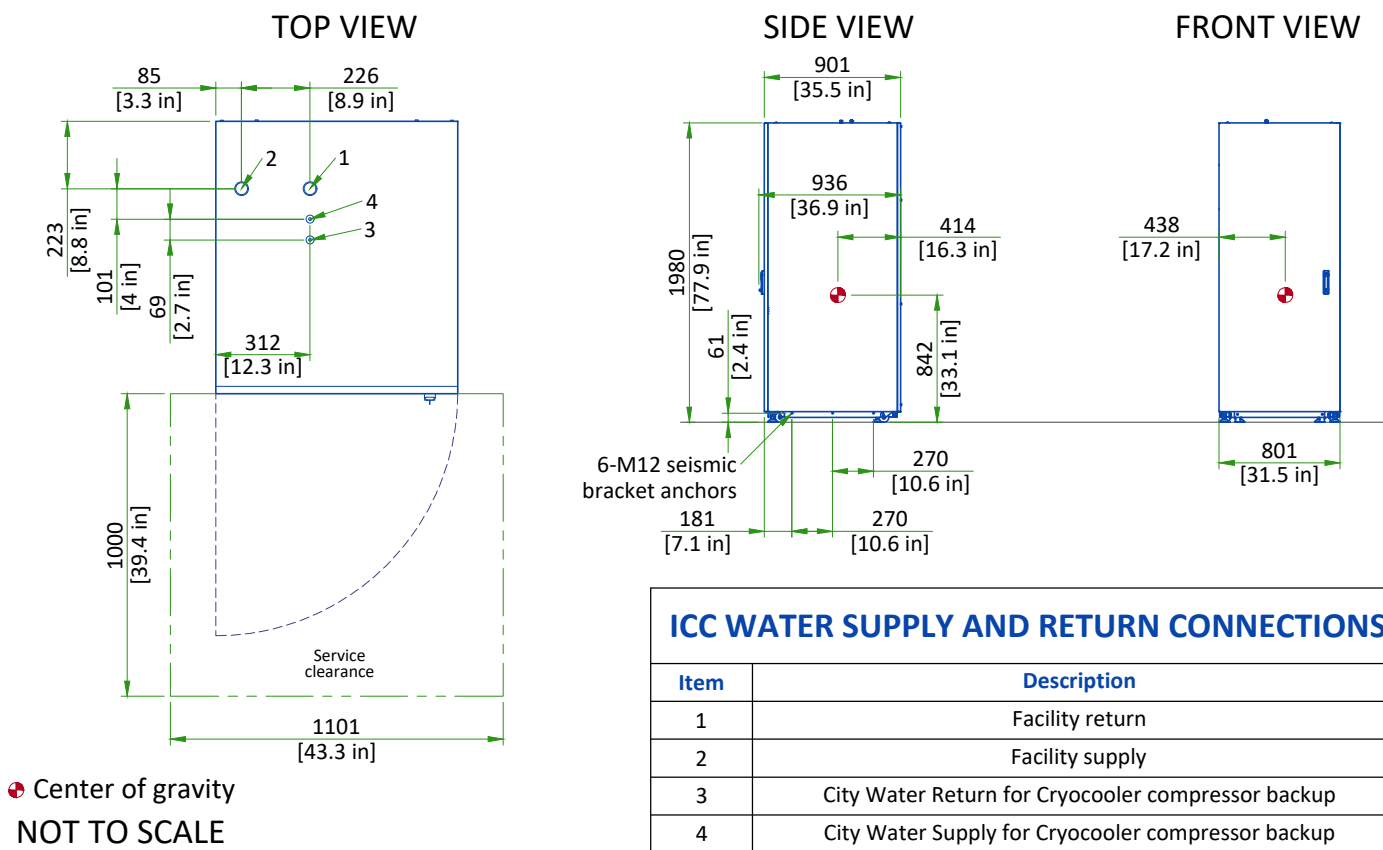


ICC WATER SUPPLY AND RETURN CONNECTIONS	
Item	Description
1	Facility return
2	Facility supply
3	City Water Return for Cryocooler compressor backup
4	City Water Supply for Cryocooler compressor backup

MAGNET MONITOR AND MAGNET RUNDOWN UNIT (MRU)

MAGNET MONITOR

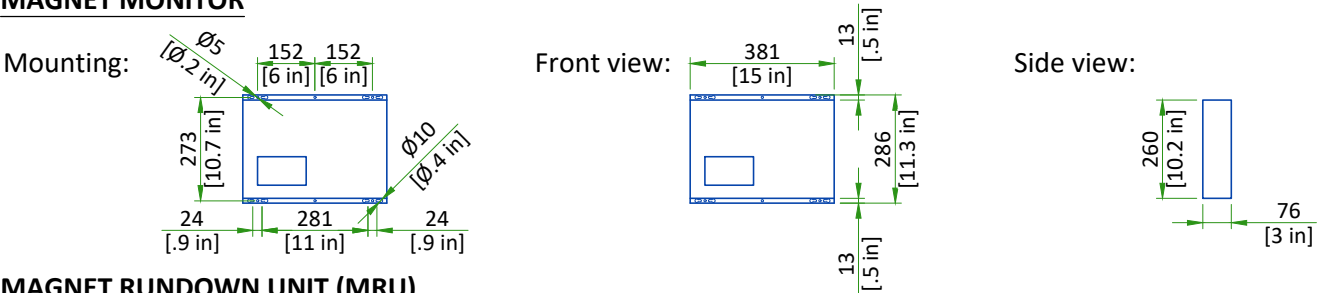

MAGNET RUNDOWN UNIT (MRU)

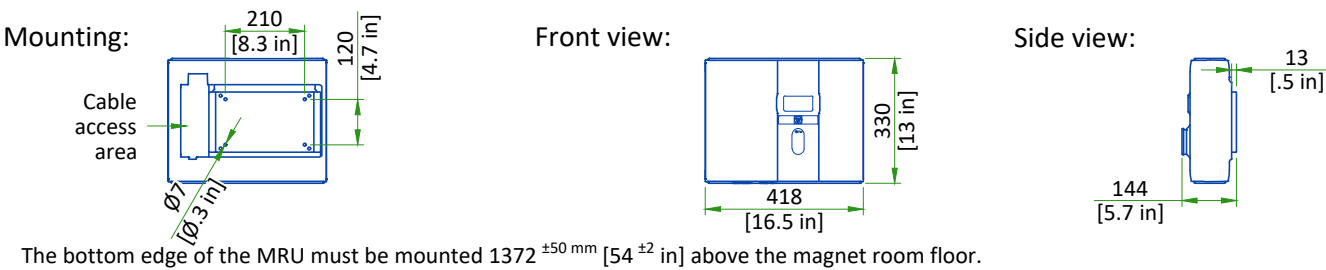

The bottom edge of the MRU must be mounted 1372 ^{±50} mm [54 ^{±2} in] above the magnet room floor.

SCALE 1:20


DELIVERY

ROUTING


- The customer is solely liable for routing of components from dock to final site.
- GEHC must be able to move system components in or out with no need to uncrate or disassemble any of the components. The entire passageway must be cleared, adequately lighted and free from dust.
- The floor and its surfacing must be able to withstand the live load of components and handling equipment.
- Floor surfacing must be continuous.
- The customer must protect any fragile flooring surfaces.

MINIMUM SPECIFICATIONS FOR MAGNET ROUTING

- Floor must be able to withstand a moving load of 4725 kg [10417 lb]
- Recommended opening height: 2.5m [98.5in], width: 2.5m [98.5in]. If recommended dimensions cannot be met refer to pre-installation manual for detailed specifications.
- Maximum slope: 30°

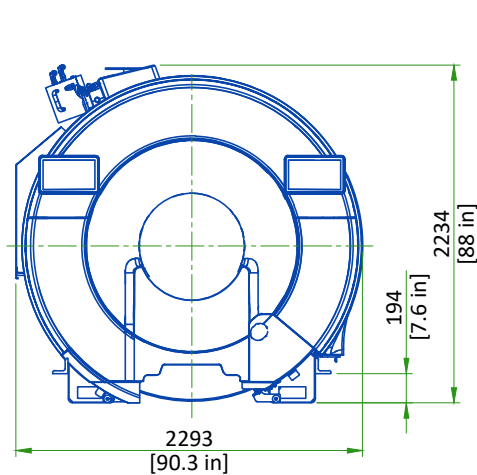


FRONT VIEW OF MAGNET

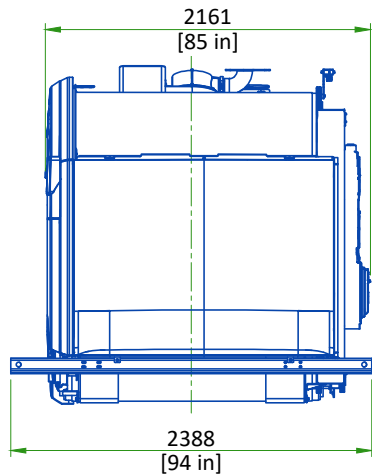


RIGHT SIDE VIEW OF MAGNET

STRAIGHT PATH

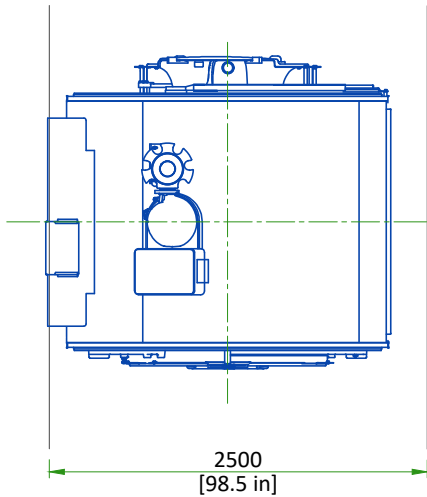

PATH WITH 90 DEGREE TURN

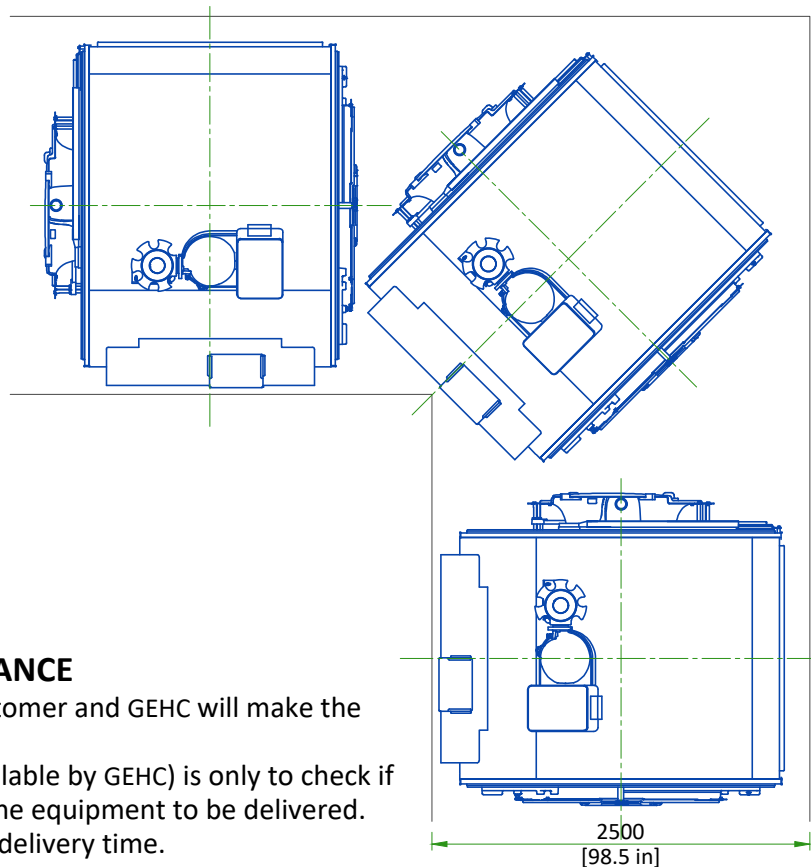

INSTALLATION AND DELIVERY ACCEPTANCE


- A survey of the site established by the customer and GEHC will make the decision for the delivery time.
- This survey of the site (a form is made available by GEHC) is only to check if the apparent conditions of the site allow the equipment to be delivered.
- If the site is not ready, GEHC can delay the delivery time.

CRITICAL ITEMS FOR MAGNET DELIVERY

- 24/7 chilled water and three phase power for Cryocooler Compressor. Refer to Power Requirements detail for detailed specifications.
- 24/7 single phase power for the Magnet Monitor. Refer to Power Requirements detail for detailed specifications.
- Ethernet connections for magnet monitoring and phone lines for emergency use. Refer to Connectivity Requirements detail for additional information.
- Operational magnet room exhaust fan. Refer to Magnet Room Venting Requirements detail for detailed specifications.
- The Cryogen Vent system is installed, complete to outside the building and ready for connection to magnet or will be completed by RF cage closure. Connection delay not to exceed 24 Hours.

This is only a partial list of items required for delivery of the magnet. For a complete checklist refer to the Pre-Installation Manual (PIM) referenced on the cover sheet.

GRADIENT COIL REPLACEMENT

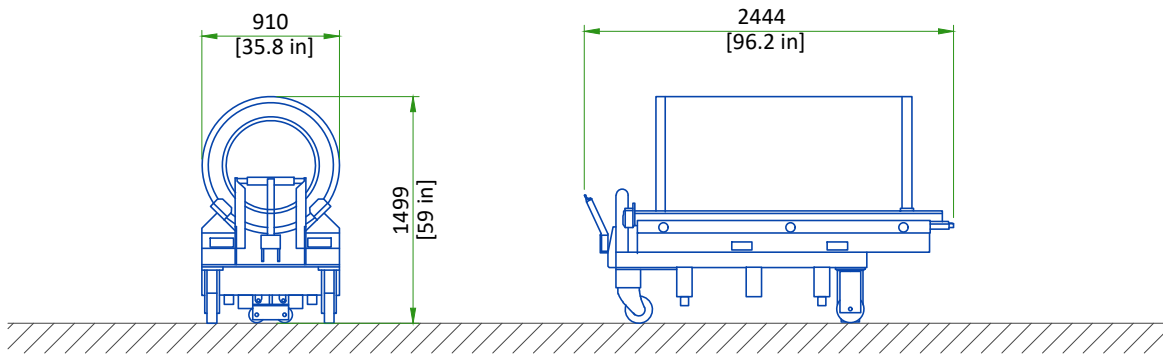

EQUIPMENT	DIMENSIONS LxWxH		WEIGHT		NOTE
	mm	in	kg	lbs	
Replacement VRMw gradient coil assembly on a shipping cradle/cart	910x2444x1499	35.8x96.2x59	1449	3194	Initial gradient coil assembly is shipped installed in the magnet. Shipping/installation cart is used to install replacement coil assembly only.

The weight bearing structure of the site should support any additional weight of the main replacement parts occurring during maintenance of the magnet, throughout the whole lifecycle of the MR.


STRUCTURAL NOTES

- All units that are wall mounted or wall supported are to be provided with supports where necessary. Wall supports are to be supplied and installed by the customer or his contractors.
- Dimensions are to finished surfaces of room.
- Certain MR procedures require an extremely stable environment to achieve high resolution image quality. Vibration is known to introduce field instabilities into the imaging system. The vibration effects on image quality can be minimized during the initial site planning of the mr suite by minimizing the vibration environment. See [PROXIMITY LIMITS](#), [PATIENT TABLE DOCK ANCHOR MOUNTING REQUIREMENTS AND VIBROACOUSTIC DAMPENING KIT](#) details for additional information.
- Standard steel studs, nails, screws, conduit, piping, drains and other hardware are acceptable if properly secured. Any loose steel objects can be violently accelerated into the bore of the magnet. Careful thought should be given to the selection of light fixtures, cabinets, wall decorations, etc. To minimize this potential hazard. For safety, all removable items within the magnet room such as faucet handles, drain covers, switch box cover plates, light fixture components, mounting screws, etc. must be non-magnetic. If you have a specific question about material, bring it to the attention of your GEHC project manager of installations.
- Floor levelness refer to [MAGNET ROOM FLOOR SPECIFICATIONS DETAIL](#), this floor levelness requirement is important for accurate patient table docking.
- Non-movable steel such as wall studs or HVAC components will produce negligible effect on the active shield magnet.
- Customer's contractor must provide all penetrations in post tension floors.
- Customer's contractor must provide and install any non-standard anchoring. Documents for standard anchoring methods are included with GEHC equipment drawings for geographic areas that require such documentation.
- Customer's contractor must provide and install hardware for "through the floor" anchoring and/or any bracing under access floors. This contractor must also provide floor drilling that cannot be completed because of an obstruction encountered while drilling by the GEHC installer such as rebar etc.
- Customer's contractor to provide and install appropriate supports for the storage of excess cables.
- It is the customer's responsibility to perform any floor or wall penetrations that may be required. The customer is also responsible for ensuring that no subsurface utilities (e.g., electrical or any other form of wiring, conduits, piping, duct work or structural supports (i.e. post tension cables or rebar)) will interfere or come in contact with subsurface penetration operations (e.g. drilling and installation of anchors/screws) performed during the installation process. To ensure worker safety, GEHC installers will perform surface penetration operations only after the customer's validation and completion of the "GEHC surface penetration permit"


VIBRATION SPECIFICATIONS


Excessive vibration can affect MR image quality. Vibration testing must be performed early in the site planning process to ensure vibration is minimized. Both steady state vibration (exhaust fans, air conditioners, pumps, etc.) and transient vibrations (traffic, pedestrians, door slamming, etc.) must be assessed. The magnet cannot be directly isolated from vibration. Any vibration issue must be resolved at the source.

Transient vibration levels above the specified limits in the MR Site Vibration Test Guidelines must be analyzed. Any transient vibration that causes vibration to exceed the steady-state level must be mitigated.

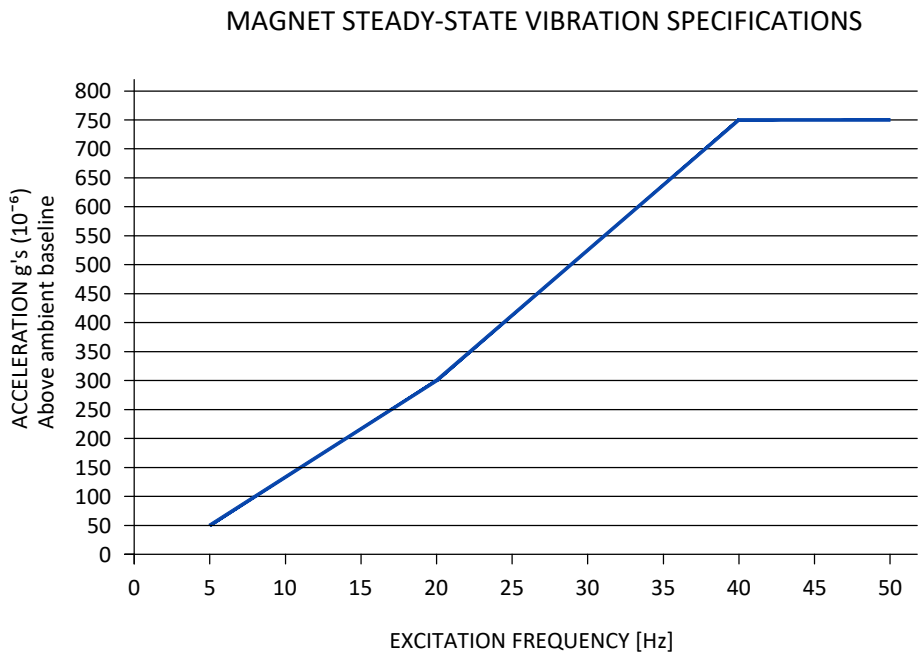


VIBRATION TRANSMITTED THROUGH VIBROACOUSTIC MAT

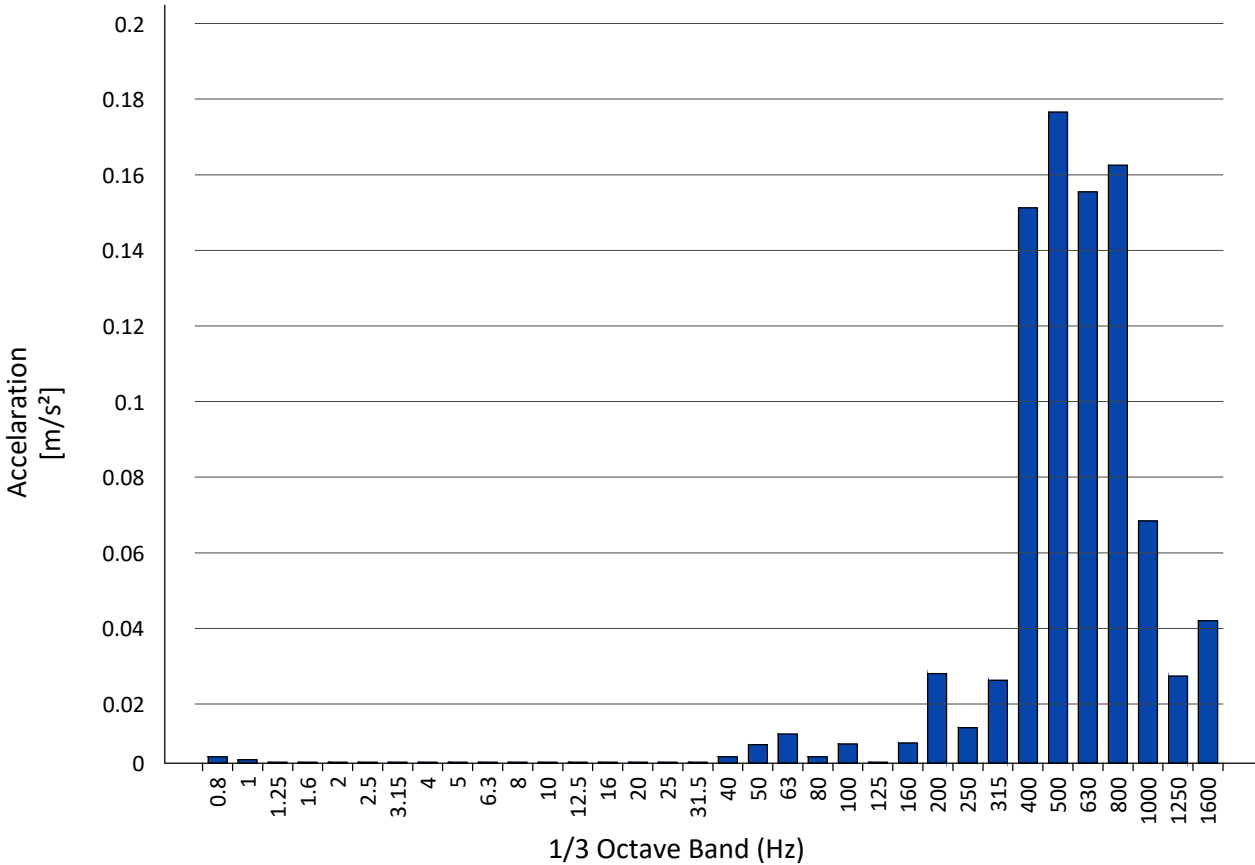

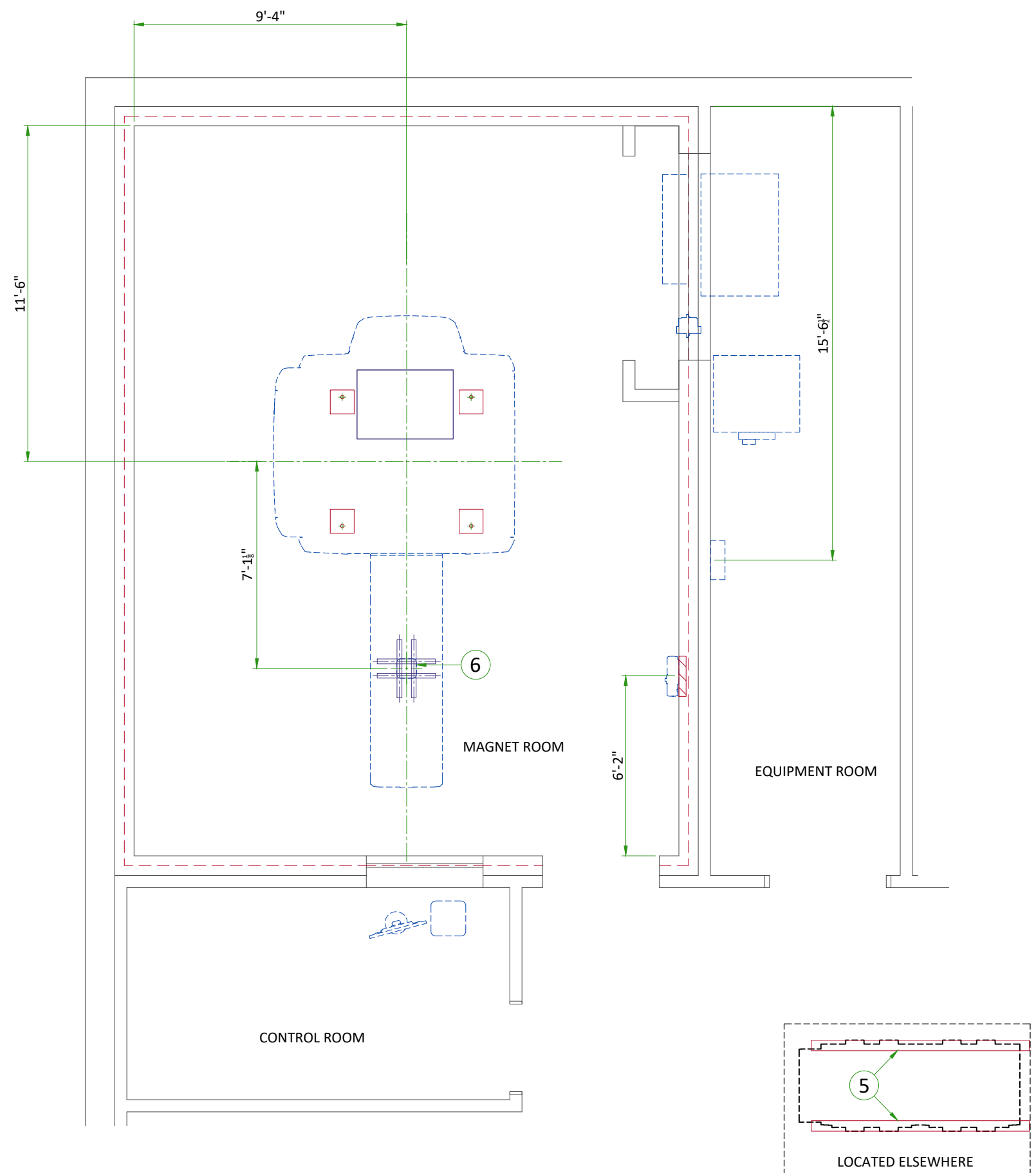

ITEM	DESCRIPTION
(GE SUPPLIED / CONTRACTOR INSTALLED)	
1	Vibroacoustic dampening kit (see floor structural detail)
2	Opening in ceiling for cable connections
(CONTRACTOR SUPPLIED & INSTALLED)	
3	Structural wall backing for LOTO Service Disconnect (FD)
4	Structural wall backing for Magnet Rundown Unit
5	Two I-beam runners are to support chiller feet at both ends. Refer to Dimplex manual.
6	Mounting location for In-room Camera option


MAGNET MOUNTING

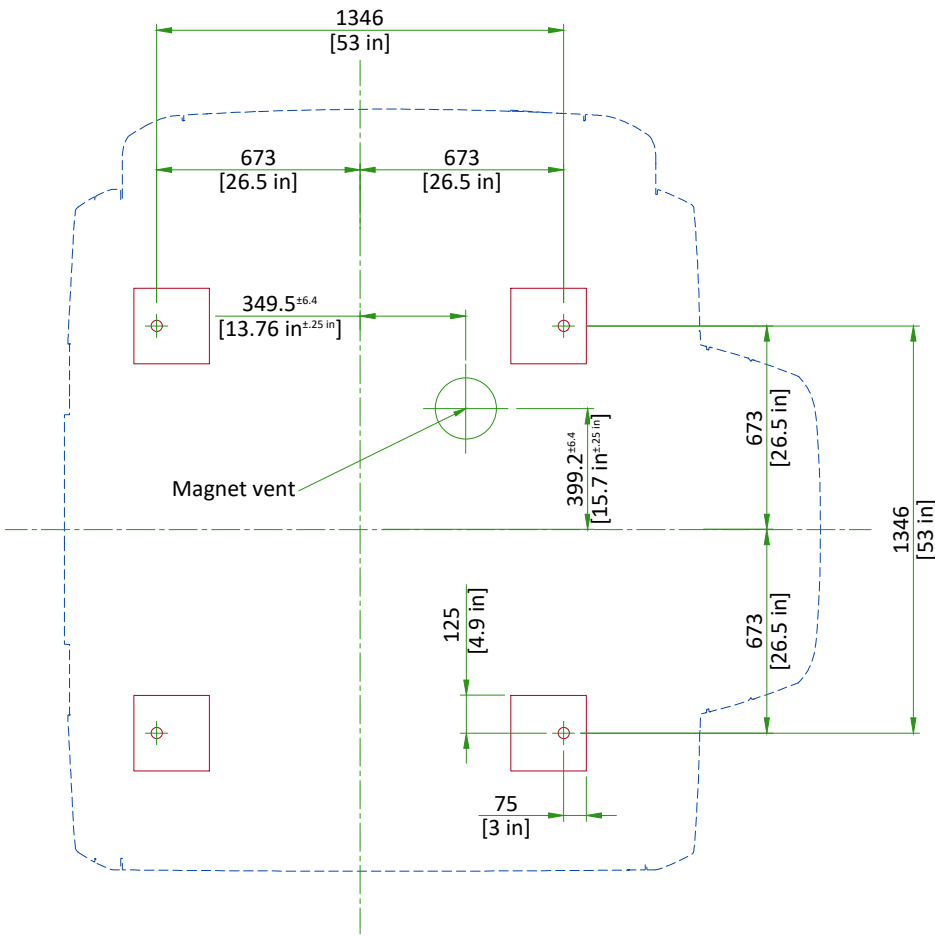

SCALE 1:25

MAGNET ROOM FLOOR SPECIFICATIONS

Magnet, Enclosure, and Patient Table areas must be flat and level within **3 mm (0.125 in)** within the shaded area shown.

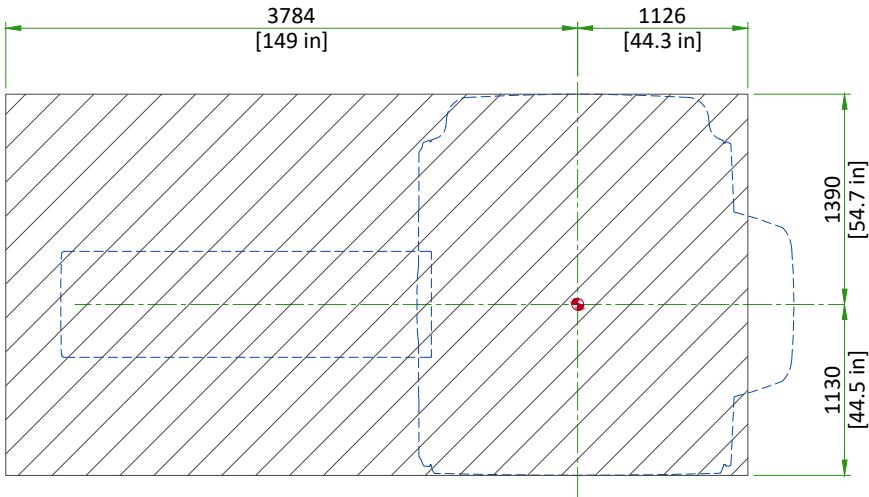


The finished floor must support the weight of all components (e.g., patient table, gradient coil replacement cart) throughout operation and service life.

SCALE 1:50

MINIMUM MAGNET CEILING HEIGHT (TOP VIEW)

If the ceiling height is between 2500 mm [98.5 in] and 2667 mm [105 in], the flexible main lead extension for low ceiling height (2.5M Low Ceiling Kit-Passive, M7000GM) is required for ramping the magnet. Contact the GE HealthCare PMI and GE HealthCare Service Field Engineer for further evaluation.

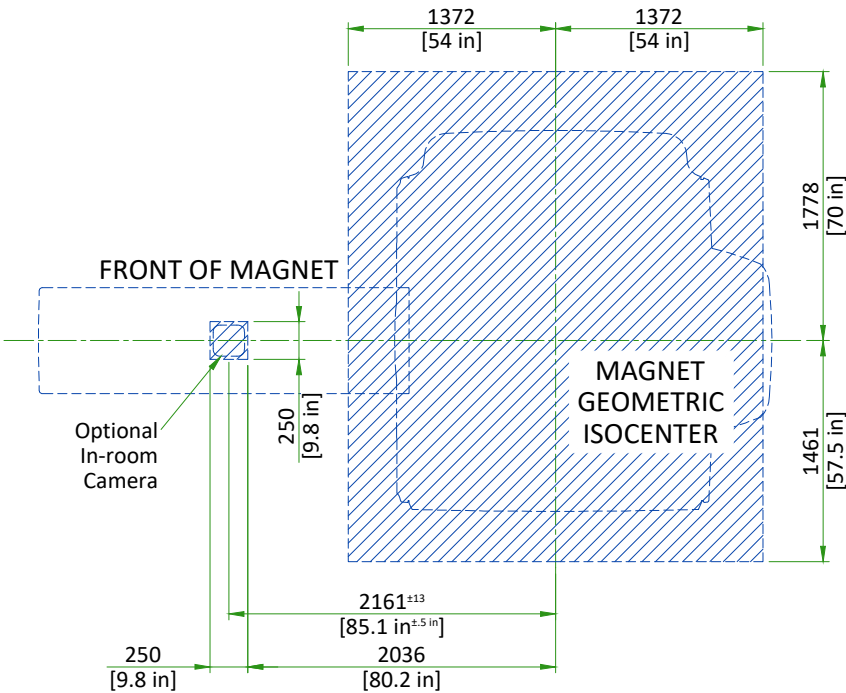


SCALE 1:50

CEILING OPENING FOR CABLE CONCEALMENT


NOTES:

- 1) The space above the cable concealment opening must be clear of any obstructions, up to a height of 2794 mm [110 in] from the finished floor, for cables to be routed to the magnet.

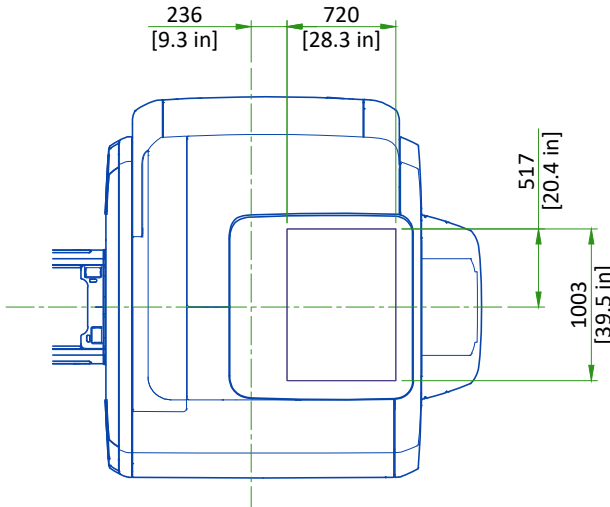


NOT TO SCALE

IN-ROOM CAMERA INSTALLATION POSITION

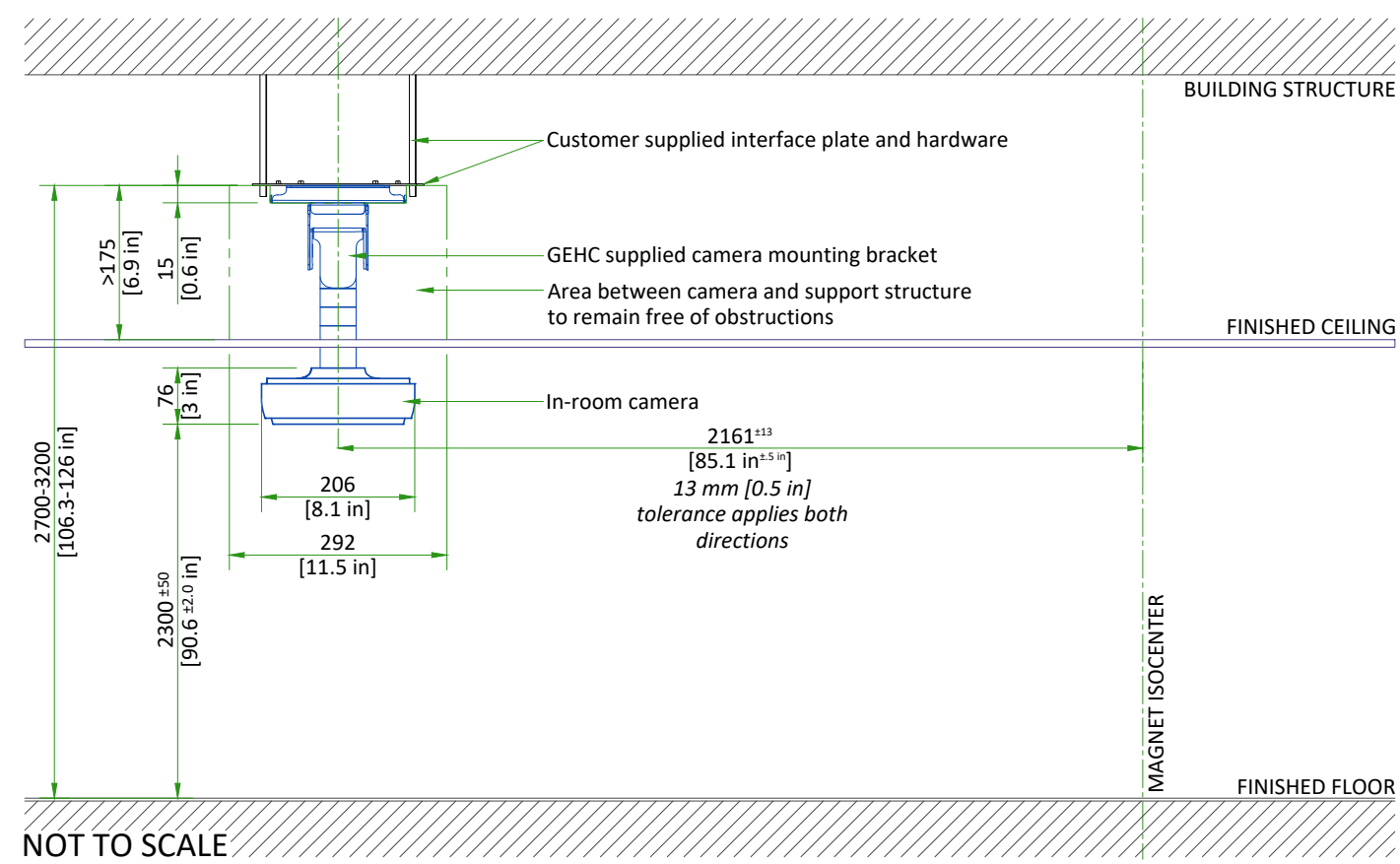


IN-ROOM CAMERA MOUNTING PLATE

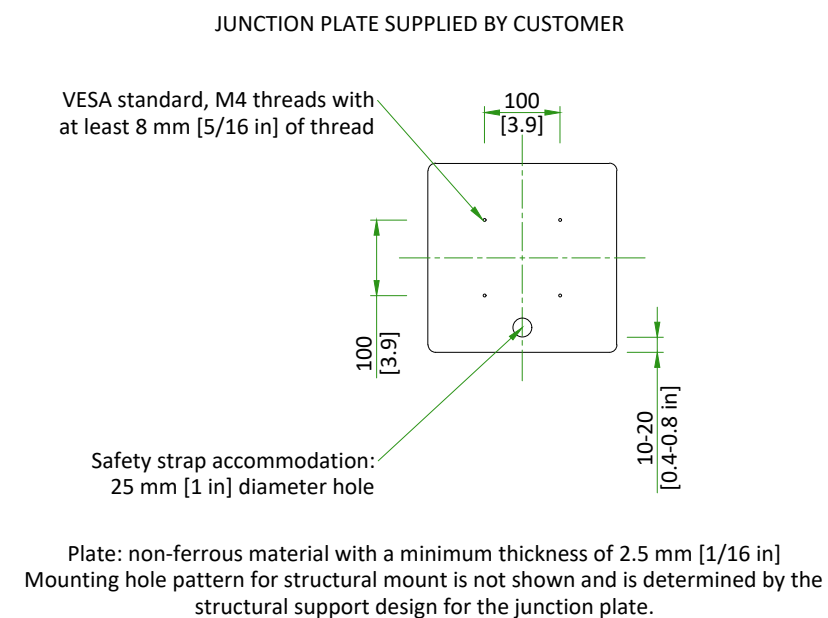

GE HealthCare will provide a mounting bracket that will interface to the customer-supplied VESA 100 x 100 junction plate using four M4 screws. The customer's architect is responsible for the design and installation of the junction plate with sufficient strength and margin to hold the camera assembly. The weight of the camera assembly is approximately 3.75 kg [8.3 lbs]. The suggested safety load on the junction plate is no less than 20 kg [44 lbs]. The junction plate should also have a 25 mm [1 in] diameter hole between 10 and 20 mm [0.4 and 0.8 in] from the plate edge. The hole will be used to attach a safety strap. Ensure that the hole does not have any sharp edges.

NOT TO SCALE

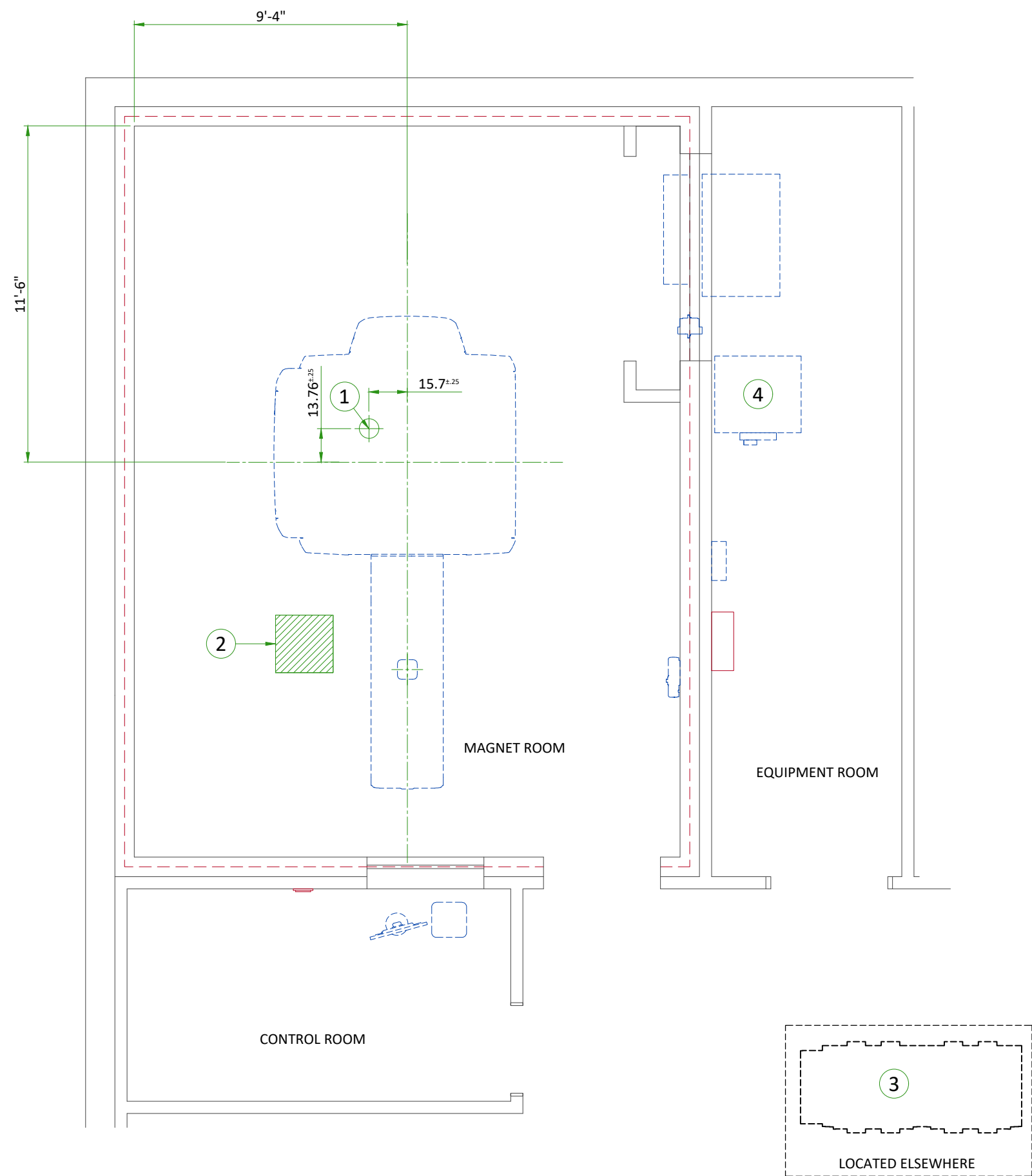

ITEM	DESCRIPTION
1	Cryogen vent (200mm [8"] O.D.)
2	Emergency exhaust vent - refer to magnet room vent requirements (position to be defined)
3	(2) 50mm [2"] I.D. High pressure hoses
4	25.4 mm [1 in] minimum hose inside diameter. If the hose is longer than 10 m [32.8 ft] and shorter than 30 m [98.4 ft], a hose with a 31.75 mm [1.25 in] minimum ID is recommended. If using 31.75 mm [1.25 in] hose, an adaptor is required to reduce the inner diameter to 25.4 mm [1 in] for the ICC connection. Refer to chilled water block diagram.

MECHANICAL/PLUMBING NOTES

- All piping, fittings, supports, hoses, clamps, ventilation systems, etc. are to be supplied and installed by the customer or his contractors.
- For complete design and requirements, specifications and guidelines refer to the pre-installation manual: system cooling, cryogen venting, waveguides and exhaust venting.
- An emergency water cooling back-up supply is recommended for continuous cryogen compressor operation. if using an open loop back-up design, ensure a drain is provided. please refer to the pre-install manual for optional back-up coolant supply requirements

TEMPERATURE AND HUMIDITY REQUIREMENTS

Room	Temperature		Humidity				
	Range		Change ¹		Absolute Humidity Ratio (Saturation Mixing Ratio) g/kg	Range %RH	Change %RH/Hr ²
	°C	°F	°C/Hr	°F/Hr			
Equipment room (at inlet to Equipment room)	15-32 ³	59-89.6 ³	3	5	≤ 11	30-75	5
Magnet Room	15-21	59-69.8	3	5	N/A	30-60	5
Operator Room	15-32	59-89.6	3	5	N/A	30-75	5
Area for patient table operation or storage	15-32 ³	59-89.6 ³	3	5	N/A	30-75	5

- NOTE
- Operating temperature gradient limits shall be between -3° C/Hr [-5° F/Hr] and 3° C/Hr [5° F/Hr], when averaged over 1 hour
 - Operating humidity gradient limits shall be between -5% RH/hour and 5% RH/hour, when averaged over 1 hour
 - Maximum ambient temperature is derated by 1° C [1.8°F] per 175 m [574 ft] above 950 m [3117 ft] (not to exceed 3000 m [9843 ft])

AIR RENEWAL
According to local standards.

NOTE
In case of using air conditioning systems that have a risk of water leakage it is recommended not to install it above electric equipment or to take measures to protect the equipment from dropping water.

MAGNET ROOM VENTING REQUIREMENTS


- HVAC VENT REQUIREMENTS
- HVAC vendor must comply with Magnet room temperature and humidity specifications and RF shielding specifications.
 - RF Shield vendor must install open pipe or honeycomb HVAC waveguides.
 - All serviceable parts in the Magnet room (e.g.: diffusers) must be non-magnetic.
 - Waveguides must be nonmagnetic and electrically isolated.
 - Incoming air must contain at least **5% air** from outside the Magnet room (inside or outside the facility) to displace residual helium.
- EMERGENCY VENT REQUIREMENT
- Exhaust vent system is supplied by the customer.
 - All items within the RF enclosure must be non-magnetic.
 - The exhaust vent system must be tested and operational before the magnet is installed.
 - The exhaust intake vent must be located near the magnet cryogenic vent at the highest point on the finished or drop ceiling.
 - The Magnet room exhaust fan and exhaust intake vent must have a capacity of at least **1200 CFM (34 m³/min)** with a minimum of **12 room air exchanges per hour**.
 - The exhaust fan must be placed above RF shielding located outside 10 gauss (1mT) and with appropriate waveguide.
 - The system must have a manual exhaust fan switch near the Operator Workspace and in the Magnet room near the door (the switches must be connected in parallel).
 - All system components must be accessible for customer inspection, cleaning and maintenance

HEAT DISSIPATION DETAILS


DESCRIPTION	ROOM	IDLE		AVERAGE		MAX	
		W	BTU/hr	W	BTU/hr	W	BTU/hr
Magnet (MAG) and Patient Table (PT)	Magnet	561	1913	1200	4095	2891	9865
Off the wall Penetration Panel (PP)	Equipment	0	0	150	512	300	1023
On the wall Penetration Panel (PP)	Equipment	0	0	0	0	0	0
Integrated System Cabinet (ISC)	Equipment	725*	2474*	2130*	7270*	7950	27125
Integrated Cooling Cabinet (ICC)	Equipment	0*	0*	500*	1706*	1000	3412
Cryocooler Compressor (CRY - Inside ICC)	Equipment	600	2047	800	2730	1000	3412
Magnet Monitor (MON)	Equipment	60	205	60	205	60	205
Operator Console equipment (CAB)	Control	60	205	60	205	60	205
OPTIONS							
MR Elastography (MRE)	Equipment	141	480	141	480	141	480
Smart Subscription Server ML350G10	Control/IT Room	1798	6134	1798	6134	1798	6134
Multi-Nuclear Spectroscopy Cabinet (MNS)	Equipment					9600	32767

*Average value is calculated using Standby mode and the Idle value is calculated using Night Mode (Low Power Mode).

MAGNET ROOM EXHAUST FAN SCHEMATIC

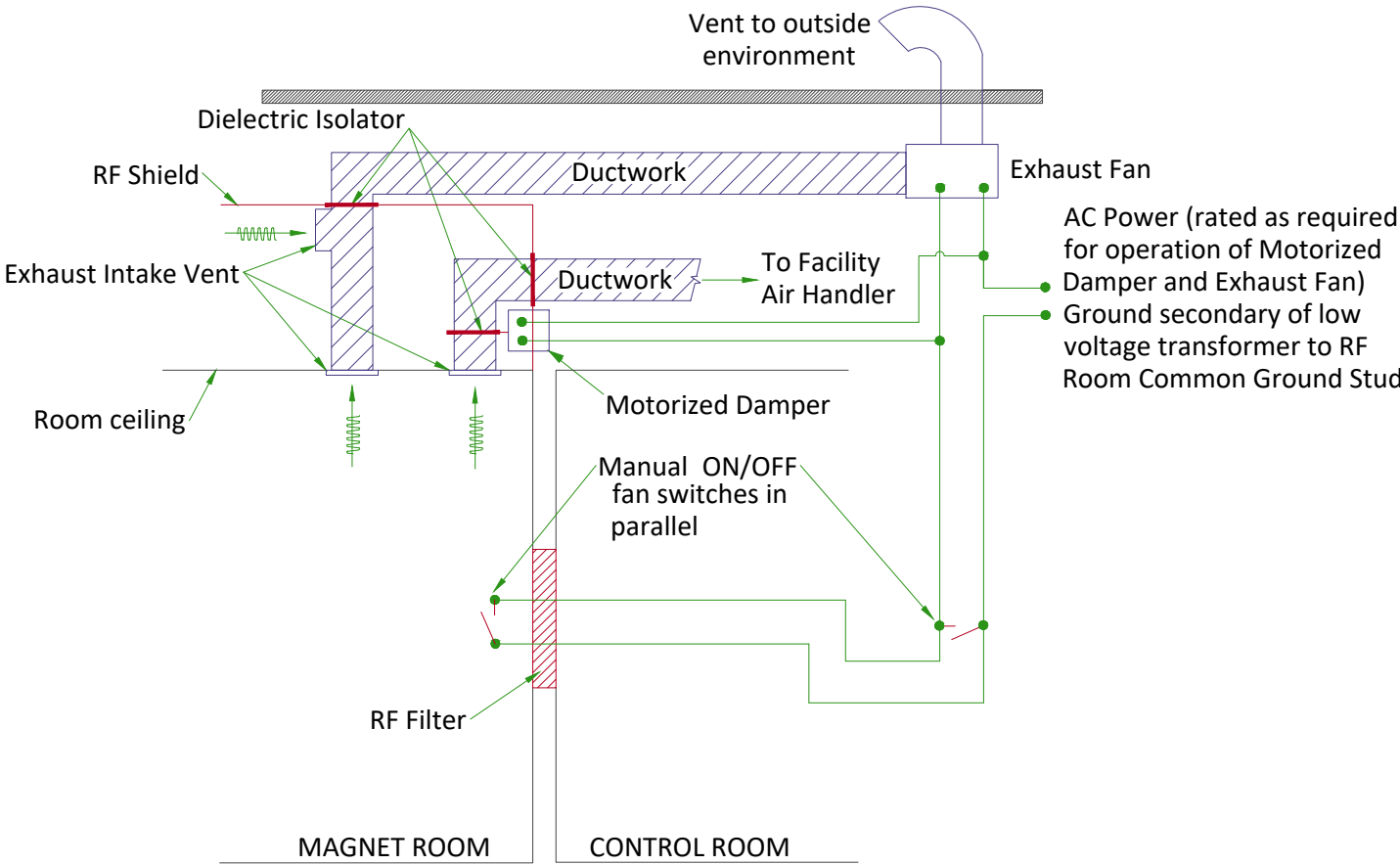

CHILLED WATER BLOCK DIAGRAM

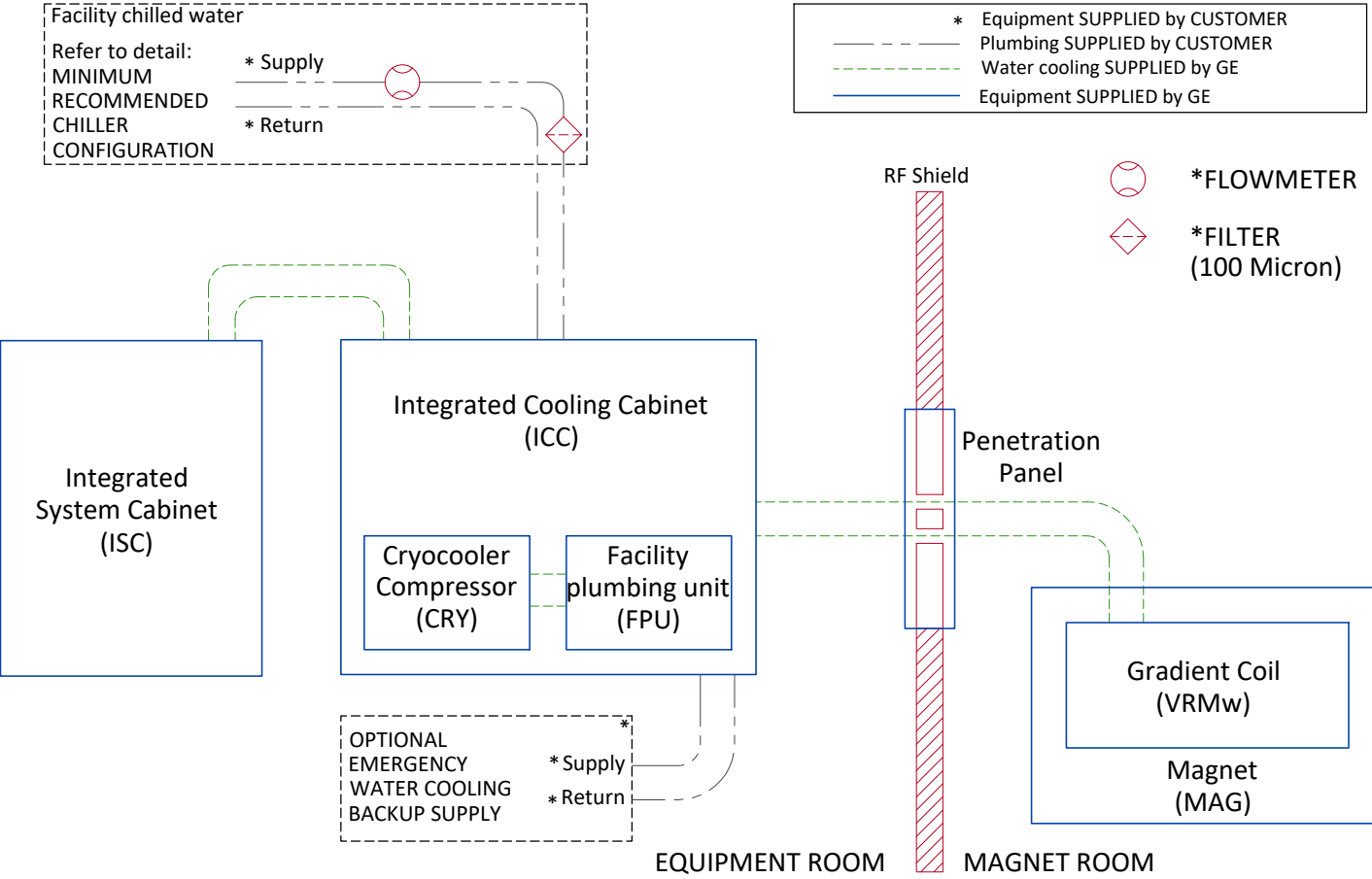


FACILITY LIQUID COOLANT REQUIREMENTS

PARAMETER	REQUIREMENTS
Availability	Continuous
Antifreeze	No more than 50% propylene glycol-water (PGW) or ethylene glycol-water (EGW)
Minimum Flow	50 L/min [13.2 gpm]
Maximum Flow	80 L/min [21.1 gpm]
Typical Flow	60 L/min [15.8 gpm]
Maximum Pressure Drop in ICC at Minimum Flow	1.8 bar [26.1 psi] with 50% PGW, 1060 kg/m ³ [66.2 lbs/ft ³] density 1.3 bar [18.9 psi] with pure water, 994 kg/m ³ [62.1 lbs/ft ³] density
Maximum Pressure Drop in ICC at Maximum Flow	3.4 bar [49.3 psi] with 50% PGW, 1060 kg/m ³ [66.2 lbs/ft ³] density 2.7 bar [39.2 psi] with pure water, 994 kg/m ³ [62.1 lbs/ft ³] density
Temperature rise at Minimum Flow	11°C [19.8°F] with 50% propylene glycol-water; 3346 J/(kg K) specific heat; 1060 kg/m ³ [66.2 lbs/ft ³] density; 32 kW heat
Temperature rise at Maximum Flow	7.0°C (12.6°F) with 50% propylene glycol-water; 3346 J/(kg K) specific heat; 1060 kg/m ³ [66.2 lbs/ft ³] density; 32 kW heat
Maximum inlet pressure to ICC	6 bar [87 psi]
Chiller size	Minimum 32 kW
Condensation protection	Facility plumbing to the ICC must be properly routed and insulated to prevent equipment damage or safety hazards
Minimum continuous heat load	7.5 kW
Inlet temperature to ICC	5 to 15°C [41 to 59° F]
Customer supplied feeder hose (from main water supply to ICC)	25.4 mm [1 in] minimum hose inside diameter
Hose connection to the ICC	38.1 mm [1.5 in] male NPT
Facility water quality requirements	Refer to pre-installation manual for detailed specifications
Particle size	< 100 micron (with field changeable filter)

CITY WATER BACKUP REQUIREMENTS FOR COMPRESSOR



INLET WATER FLOW/TEMPERATURE FOR CRYOCOOLER COMPRESSOR

*RECOMMENDED TEMPERATURE AND WATER FLOW TO CRY ENTRY

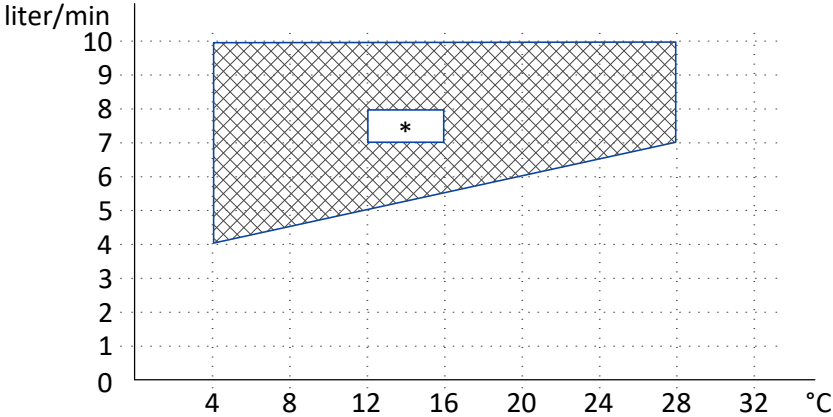


	MIN	MAX	IDEAL
INLET TEMP (°C)	4	28	12-16
INLET FLOW (l/min)	4	10	7-8
INLET PRESSURE (kPa)	200	690	
TEMP RISE	26°C at 4 l/min flow	10°C at 10 l/min flow	
HEAT DISSIPATION (kW)	7.2 kW		
PRESSURE DROP	60 kPa at 8 l/min flow		

MINIMUM RECOMMENDED CHILLER CONFIGURATION

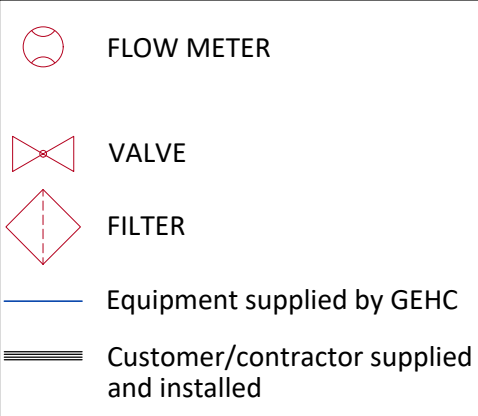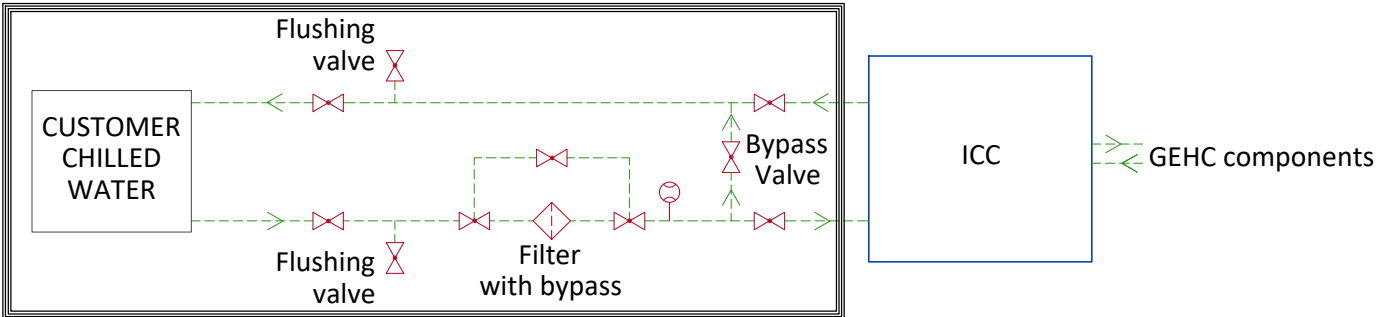

TYPICAL CRYOGENIC VENT PIPE DETAIL

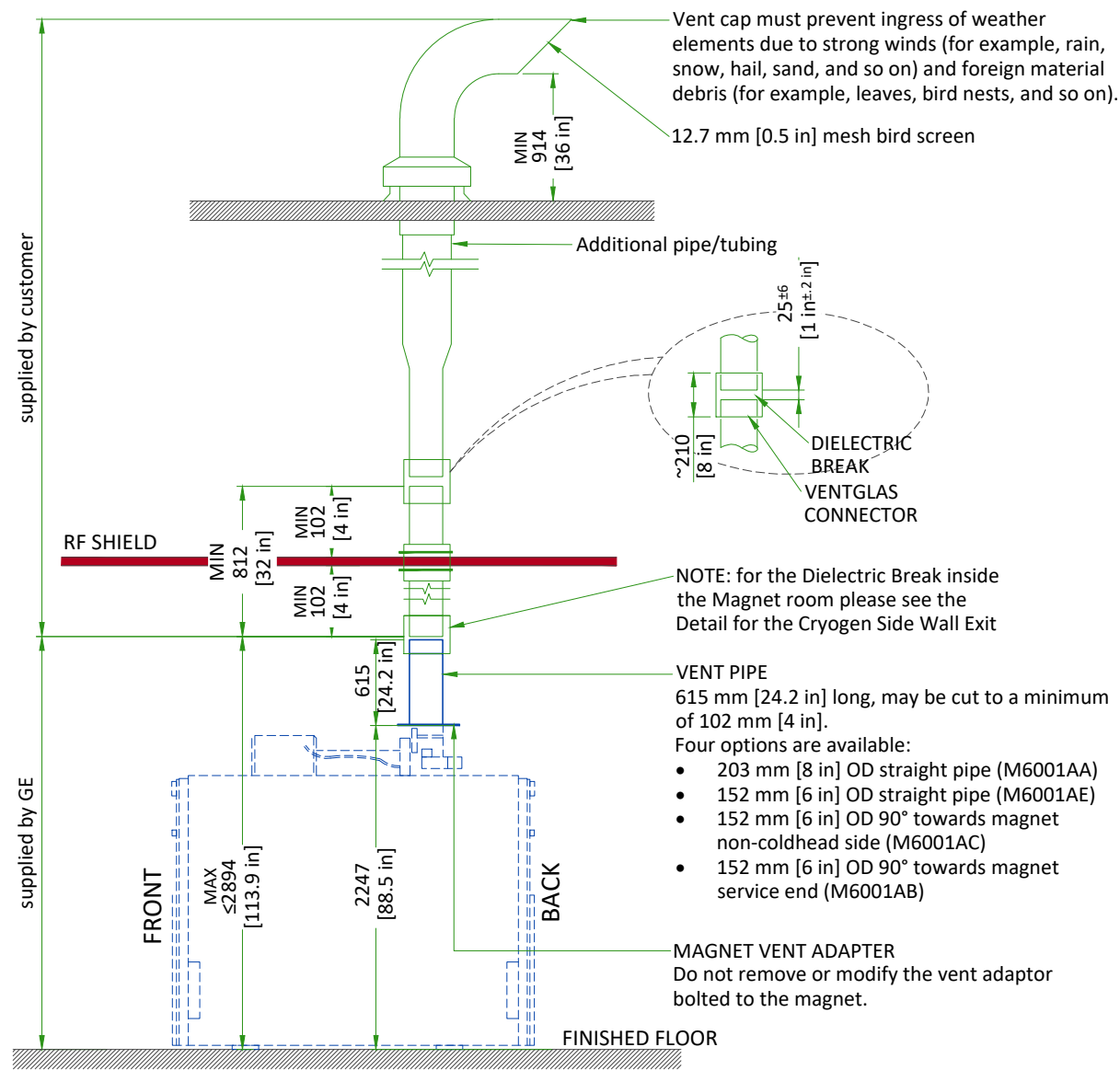

Waveguide is contractor supplied. Minimum 812 mm [32 in]. Must extend at least 102 mm [4 in] on magnet room side of the wall/ceiling and 25±6 mm [1±0.25 in] from the GEHC supplied pipe below isolation joint.

1. The 203 mm [8 in] or 152 mm [6 in] OD vent material must be one of the following materials with the wall thickness indicated:
 - a. SS 304: Minimum 0.89 mm [0.035 in]; Maximum 3.18 mm [0.125 in]
 - b. AL 6061-T6: Minimum 2.11 mm [0.083 in]; Maximum 3.18 mm [0.125 in]
 - c. CU DWV, M or L: Minimum 2.11 mm [0.083 in]; Maximum 3.56 mm [0.140 in]

NOTE


All welds on the pipe must be ground down to a smooth 203 mm [8 in] or 152 mm [6 in] diameter so that it can be clamped to the Ventglas with enough force.

3. Corrugated pipe or spiral duct must not be used
4. If required, bellows pipe less than 300 mm [12 in] in length may be used as a thermal expansion joint
5. The vent pipe must withstand the maximum pressure listed in the Pre-Installation Manual
6. Waveguide vent material must match the outside diameter of the magnet flanged vent adapter
7. The vent system must be insulated with 38 mm [1.5 in] thick flexible unicellular insulation to prevent condensation during magnet ramping. Exposed insulation must be covered with a white PVC jacket. The clearance area of the dielectric break must be left bare (without insulation) or the insulation must be easily removable for periodic inspection.


TYPICAL CRYOGEN SIDE WALL EXIT

KEY COMPONENTS:

- RF waveguide extended from wall to magnet adapter
- Must be all same material and all welded or brazed
- Support system must withstand 8229 N [1850 lbs]
- GEHC ventglas must be installed in vertical section directly over magnet
- Vent cap must prevent ingress of weather elements due to strong wind (for example, rain, snow, hail, sand, and so on) and foreign material debris (for example, leaves, bird nests, and so on).

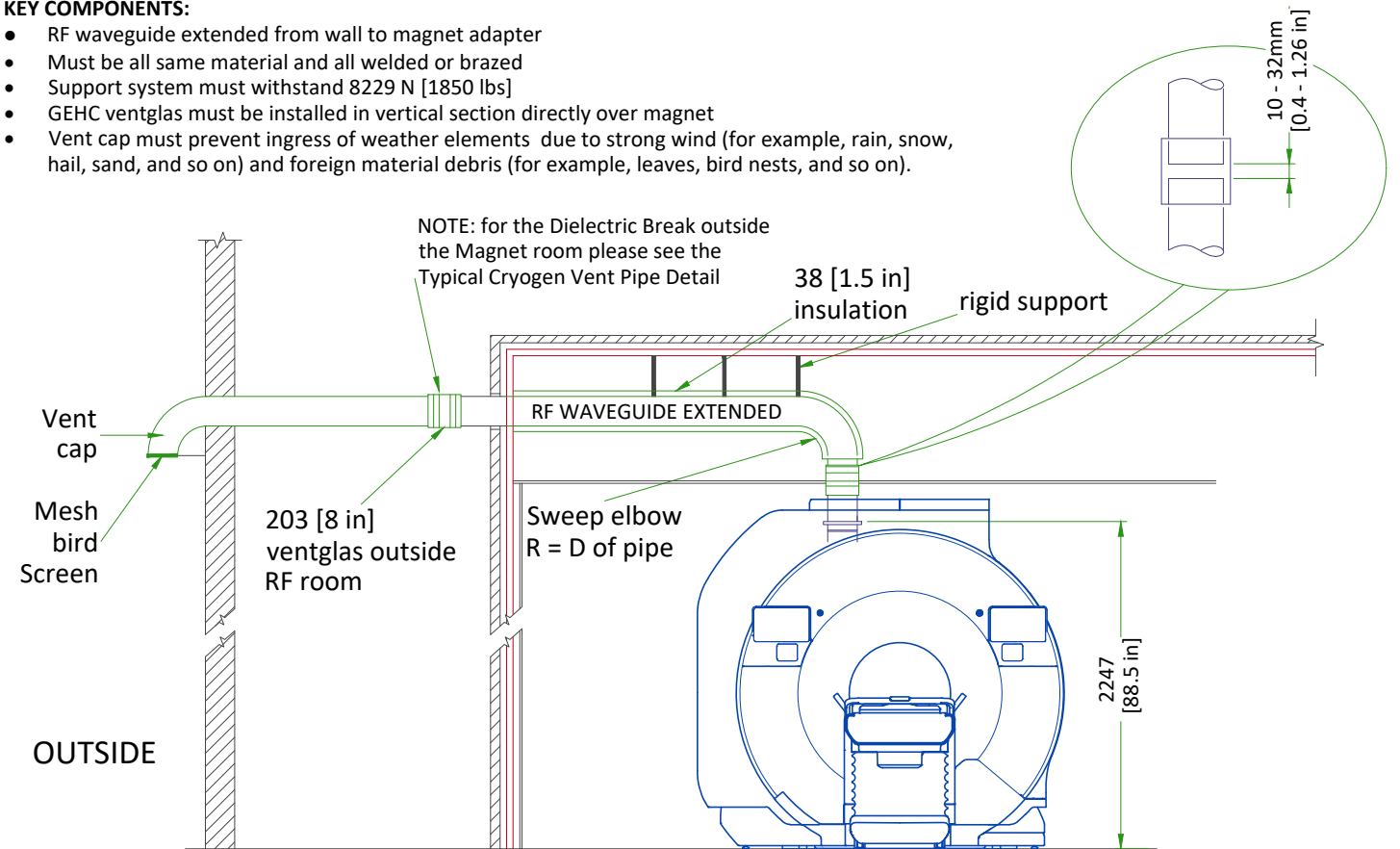

CRYOGENIC VENTING (EXTERIOR)

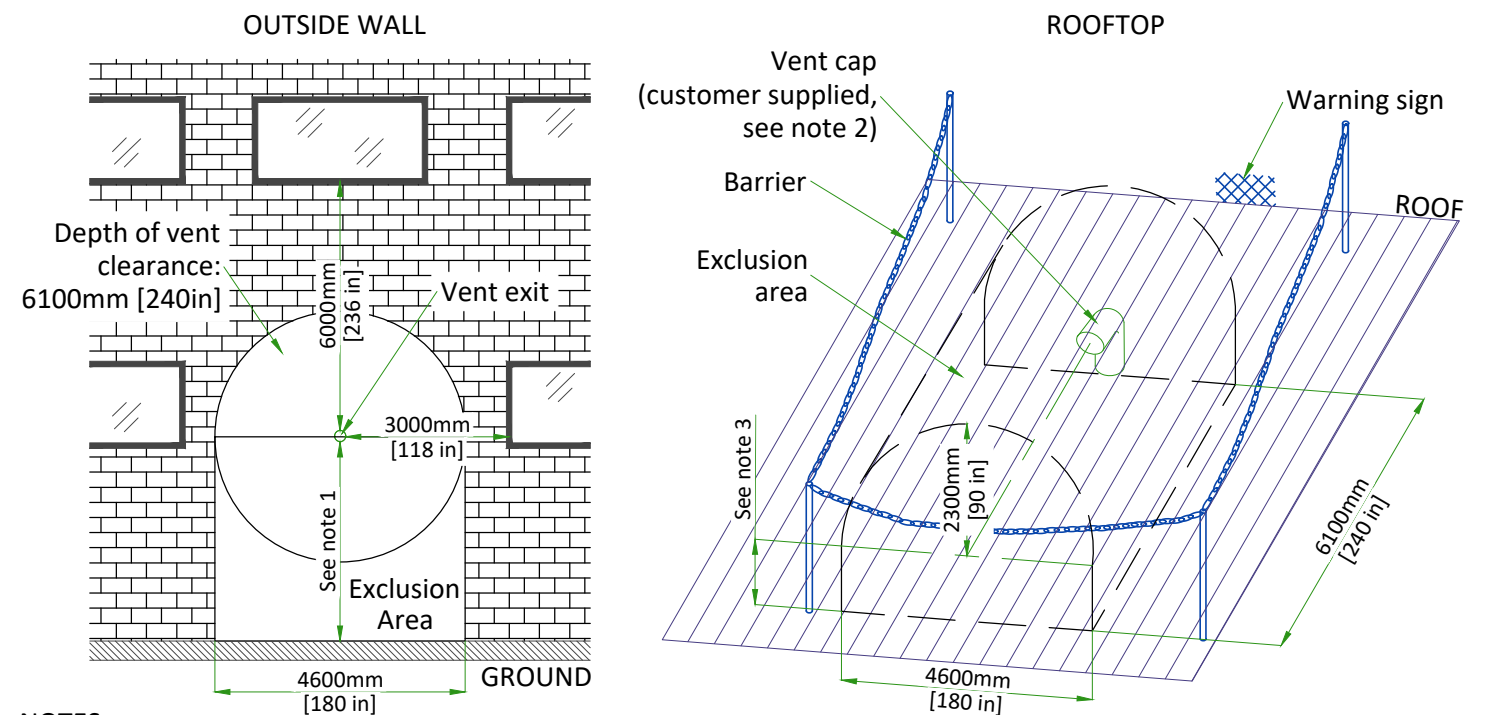

NOTES

- (1) Restricted area: minimum distance between vent pipe and ground is 3660mm [144 in]. Barriers are required. Public area: barriers are not required if height is > 5000mm [197 in].
- (2) The bottom of the 90° elbow must be at least 914mm [36 in] above the roof deck (or higher if at risk of being blocked by drifting snow, sand, or other potential obstructions.)
- (3) 2300 mm [90 in] or distance to surface of rooftop.

NOT TO SCALE

MAGNET CRYOGENIC VENT SYSTEM PRESSURE DROP MATRIX																		
Outer dia. of pipe (D)	Distance of vent system component from magnet		Pressure drop for straight pipe		Std sweep 45° elbow		Long sweep 45° elbow		Std sweep 90° elbow		Long sweep 90° elbow		90° miter bend					
	ft	m	psi/ft	kPa/m	psi	kPa	psi	kPa	psi	kPa	psi	kPa	psi	kPa				
150 mm (6 in.)	0-10	0.00-3.05	0.146	3.311	0.804	5.544	0.536	3.696	1.507	10.394	1.005	6.93	3.014	20.789				
	10-20	3.05-6.10	0.253	5.715	1.356	9.355	0.904	6.237	2.543	17.54	1.696	11.694	5.087	35.081				
	20-30	6.10-9.15	0.374	8.451	1.845	12.727	1.23	8.485	3.46	23.864	2.307	15.909	6.92	47.727				
	30-40	9.15-12.19	0.473	10.699	2.278	15.708	1.518	10.472	4.271	29.453	2.847	19.635	8.541	58.906				
	40-50	12.19-15.25	0.554	12.534	2.66	18.342	1.773	12.228	4.987	34.392	3.325	22.928	9.974	68.783				
	50-60	15.25-18.29	0.62	14.019	2.997	20.668	1.998	13.779	5.619	38.753	3.746	25.835	11.238	77.506				
200 mm (8 in.)	0-10	0.00-3.05	0.025	0.564	0.188	1.294	0.125	0.862	0.352	2.426	0.234	1.617	0.703	4.851				
	10-20	3.05-6.10	0.043	0.97	0.313	2.158	0.209	1.439	0.587	4.046	0.391	2.697	1.173	8.092				
	20-30	6.10-9.15	0.064	1.45	0.427	2.944	0.285	1.963	0.8	5.52	0.534	3.68	1.601	11.04				
	30-40	9.15-12.19	0.082	1.862	0.53	3.658	0.354	2.439	0.995	6.859	0.663	4.573	1.989	13.718				
	40-50	12.19-15.25	0.098	2.215	0.624	4.307	0.416	2.871	1.171	8.075	0.781	5.383	2.342	16.15				
	50-60	15.25-18.29	0.111	2.516	0.71	4.895	0.473	3.263	1.331	9.179	0.887	6.119	2.662	18.357				
	60-80	18.29-24.39	0.132	2.987	0.857	5.914	0.572	3.942	1.608	11.088	1.072	7.392	3.216	22.176				
	80-100	24.39-30.49	0.147	3.318	0.979	6.752	0.653	4.501	1.836	12.659	1.224	8.439	3.671	25.318				
250 mm (10 in.)	0-20	0.00 - 6.10	0.011	0.241	0.099	0.683	0.066	0.455	0.186	1.28	0.124	0.854	0.371	2.561				
	20-40	6.10-12.19	0.021	0.468	0.168	1.16	0.112	0.773	0.315	2.175	0.21	1.45	0.631	4.351				
	40-60	12.19-18.29	0.029	0.645	0.227	1.568	0.152	1.045	0.426	2.94	0.284	1.96	0.853	5.88				
	60-80	18.29-24.39	0.035	0.781	0.278	1.916	0.185	1.277	0.521	3.592	0.347	2.395	1.042	7.184				
	80-100	24.39-30.49	0.039	0.884	0.321	2.212	0.214	1.474	0.601	4.147	0.401	2.765	1.203	8.294				
300 mm (12 in.)	0-20	0.00 - 6.10	0.004	0.08	0.04	0.277	0.027	0.184	0.075	0.519	0.05	0.346	0.15	1.037				
	20-40	6.10-12.19	0.007	0.157	0.068	0.47	0.045	0.313	0.128	0.88	0.085	0.587	0.255	1.761				
	40-60	12.19-18.29	0.01	0.22	0.093	0.638	0.062	0.425	0.174	1.197	0.116	0.798	0.347	2.393				
	60-80	18.29-24.39	0.012	0.269	0.114	0.786	0.076	0.524	0.214	1.473	0.142	0.982	0.427	2.946				
	80-100	24.39-30.49	0.014	0.309	0.133	0.914	0.088	0.609	0.248	1.714	0.166	1.142	0.497	3.427				
350 mm (14 in.)	0-20	0.00 - 6.10	0.001	0.032	0.019	0.13	0.013	0.086	0.035	0.243	0.024	0.162	0.071	0.486				
	20-40	6.10-12.19	0.003	0.063	0.032	0.219	0.021	0.146	0.06	0.411	0.04	0.274	0.119	0.823				
	40-60	12.19-18.29	0.004	0.088	0.043	0.299	0.029	0.2	0.081	0.561	0.054	0.374	0.163	1.122				
	60-80	18.29-24.39	0.005	0.11	0.054	0.37	0.036	0.247	0.101	0.694	0.067	0.463	0.201	1.389				
	80-100	24.39-30.49	0.006	0.127	0.063	0.433	0.042	0.289	0.118	0.812	0.079	0.542	0.236	1.625				
400 mm (16 in.)	0-20	0.00 - 6.10	0.001	0.014	0.01	0.068	0.007	0.045	0.018	0.127	0.012	0.084	0.037	0.253				
	20-40	6.10-12.19	0.001	0.028	0.017	0.114	0.011	0.076	0.0310	0.2130	0.021	0.142	0.062	0.427				
	40-60	12.19-18.29	0.002	0.04	0.023	0.156	0.015	0.104	0.042	0.292	0.028	0.195	0.085	0.584				
	60-80	18.29-24.39	0.002	0.05	0.028	0.193	0.019	0.129	0.053	0.362	0.035	0.242	0.105	0.725				
	80-100	24.39-30.49	0.003	0.059	0.033	0.227	0.022	0.151	0.062	0.426	0.041	0.284	0.124	0.852				
Notes																		
1. Refer to Magnet Room Venting manual 5850263-1EN for specifications of distances >100 ft (30.49 m).																		
2. Elbows with angles greater than 90° must not be used																		
3. Data in Table is based on the following facts and assumptions:																		
a. Initial flow conditions at magnet interface																		
b. EM energy (13MJ) is dumped to He during quench and rises He temperature to 10 Kelvin																		
c. Gas temperature starting at 10 Kelvin and increase with length determined by thermal energy balance																		
d. 90% He is assumed to be evacuated within 30 sec. None left after quench.																		
e. Absolute roughness is assumed to be 0.25 mm.																		
f. R/D = 1.0 for standard sweep elbows, R/D = 1.5 for long sweep elbows where D = outer diameter of pipe; R = radius of bend																		
4. The total pressure drop of the entire cryogenic vent system varies with the type of adapter selected, refer to Magnet Room Venting manual. The calculation starts at the magnet vent interface and ends at the termination point outside the building.																		
----							SIGNA SPRINT SELECT			EN-MRI-TYP-SIGNA SPRINT SELECT-NF.DWG			Rev 0 Date 01/Apr/2026		M5 - Cryogenics (2)		19/26	

LIGHTING REQUIREMENTS

- All lighting fixtures and associated components must meet all RF shielded room and RF grounding requirements (e.g., track lighting is not recommended due to possible RF noise).
- All removable lighting fixtures and associated components must be non-magnetic.
- All lighting must use direct current (the DC must have less than 5% ripple).
- At least 300 lux must be provided at the front of the magnet for patient access and proper functioning of the in-room camera above the magnet for servicing.
- Fluorescent lighting must not be used in the magnet room.
- Lighting must be adjusted using a discrete switch or a variable DC lighting controller.
- SCR dimmers or rheostats must not be used.
- DC LED lighting may be used if the DC power converter and RF sources are all located outside the magnet room RF shield.

NOTE: LED lighting could cause image quality issues due to RF interference. Make sure a MR-compatible LED lighting solution is chosen.
- Battery chargers (e.g., used for emergency lighting) must be located outside the magnet room.
- LED Lighting or short filament length incandescent bulbs are recommended.
- Linear lamps are not recommended due to the high burnout rate.

CONNECTIVITY REQUIREMENTS

Your new GE Healthcare imaging modality will require local and remote connectivity to enable our full range of digital support:

- Local connectivity - This allows your system to connect to local devices such as PACS and modality worklist. We will require network information to configure the system(s), and a live ethernet port(s) prior to the delivery of the system(s).
- Remote connectivity - Your GE Healthcare service warranty includes InSite™ (applicable to InSite capable products), a powerful broadband-based service which enables digital tools that can help guard your hospital against equipment downtime and revenue loss by quickly connecting you to a GE Healthcare expert.


Depending on product family and software version, imaging systems can be connected in one of the following methods:

1. TLS over TCP Port 443 (Preferred method for new products) via:
 - a. DNS resolution
 - b. Customer-provided Proxy or
 - c. GE Proxy (Available in some regions)
2. Site-to-Site IPsec VPN tunnel

Please provide the GE project manager with the contact information for the resource that can provide information required to set up these connections. GEHC will send out communication to these contacts, which will include the project's Connectivity requirements, and a Connectivity form. This form will need to be completed and returned to GEHC prior to delivery of the system to ensure the system is tested and connectivity is enabled prior to the completion of the installation.

ELECTRICAL NOTES


1. Aluminum or solid wires are not allowed.
 2. Wire sizes given are for use of equipment. Larger sizes may be required by local codes.
 3. It is recommended that all wires be color coded, as required in accordance with national and local electrical codes.
 4. Conduit sizes shall be verified by the architect, electrical engineer or contractor, in accordance with local or national codes.
 5. Convenience outlets are not illustrated. Their number and location are to be specified by others. Locate at least one convenience outlet close to the system control, the power distribution unit and one on each wall of the procedure room. Use hospital approved outlet or equivalent.
 6. General room illumination is not illustrated. Caution should be taken to avoid excessive heat from overhead spotlights. Damage can occur to ceiling mounting components and wiring if high wattage bulbs are used. Recommend low wattage bulbs no higher than 75 watts and use dimmer controls (except MR). Do not mount lights directly above areas where ceiling mounted accessories will be parked.
 7. Routing of cable ductwork, conduits, etc., must run direct as possible otherwise may result in the need for greater than standard cable lengths (refer to the interconnection diagram for maximum usable lengths point to point).
 8. Conduit turns to have large, sweeping bends with minimum radius in accordance with national and local electrical codes.
 9. In some cases GEHC will specify ground wires to be sized larger than code. In these situations, the GEHC specification must be followed.
 10. A special grounding system is required in all procedure rooms by some national and local codes. It is recommended in areas where patients might be examined or treated under present, future, or emergency conditions. Consult the governing electrical code and confer with appropriate customer administrative personnel to determine the areas requiring this type of grounding system.
 11. The maximum point to point distances illustrated on this drawing must not be exceeded.
 12. Physical connection of primary power to GEHC equipment is to be made by customers electrical contractor with the supervision of a GEHC representative. The GEHC representative would be required to identify the physical connection location, and insure proper handling of GEHC equipment.
 13. GEHC conducts power audits to verify quality of power being delivered to the system. The customer's electrical contractor is required to be available to support this activity.
 14. Every installation is unique. The electrical contractor will be required to support the installation of the GEHC equipment by providing knockouts, grommeted openings, bushings, etc. as required. All power connections to be performed by the electrician.
-
- All junction boxes, conduit, duct, duct dividers, switches, circuit breakers, cable tray, etc., are to be supplied and installed by customers electrical contractor. All junction boxes shall be provided with covers.
 - Conduit and duct runs shall have gradual sweep radius bends.
 - Conduits and duct above ceiling or below finished floor must be installed as near to ceiling or floor as possible to reduce run length.
 - Ceiling mounted junction boxes illustrated on this plan must be installed flush with finished ceiling.
 - All ductwork must meet the following requirements:
 - 1.Ductwork shall be metal with dividers and have removable, accessible covers.
 2. Ductwork shall be certified/rated for electrical power purposes.
 3. Ductwork shall be electrically and mechanically bonded together in an approved manner.
 4. PVC as a substitute must be used in accordance with all local and national codes.
 - All openings in raceway and access flooring are to be cut out and finished off with grommet material by the customers contractor.
 - Electrical contractor to provide measured pull strings in all conduit and raceway runs.
 - Provide 10 foot pigtails at all junction points.
 - Grounding is critical to equipment function and patient safety. Site must conform to wiring specifications shown on this plan.

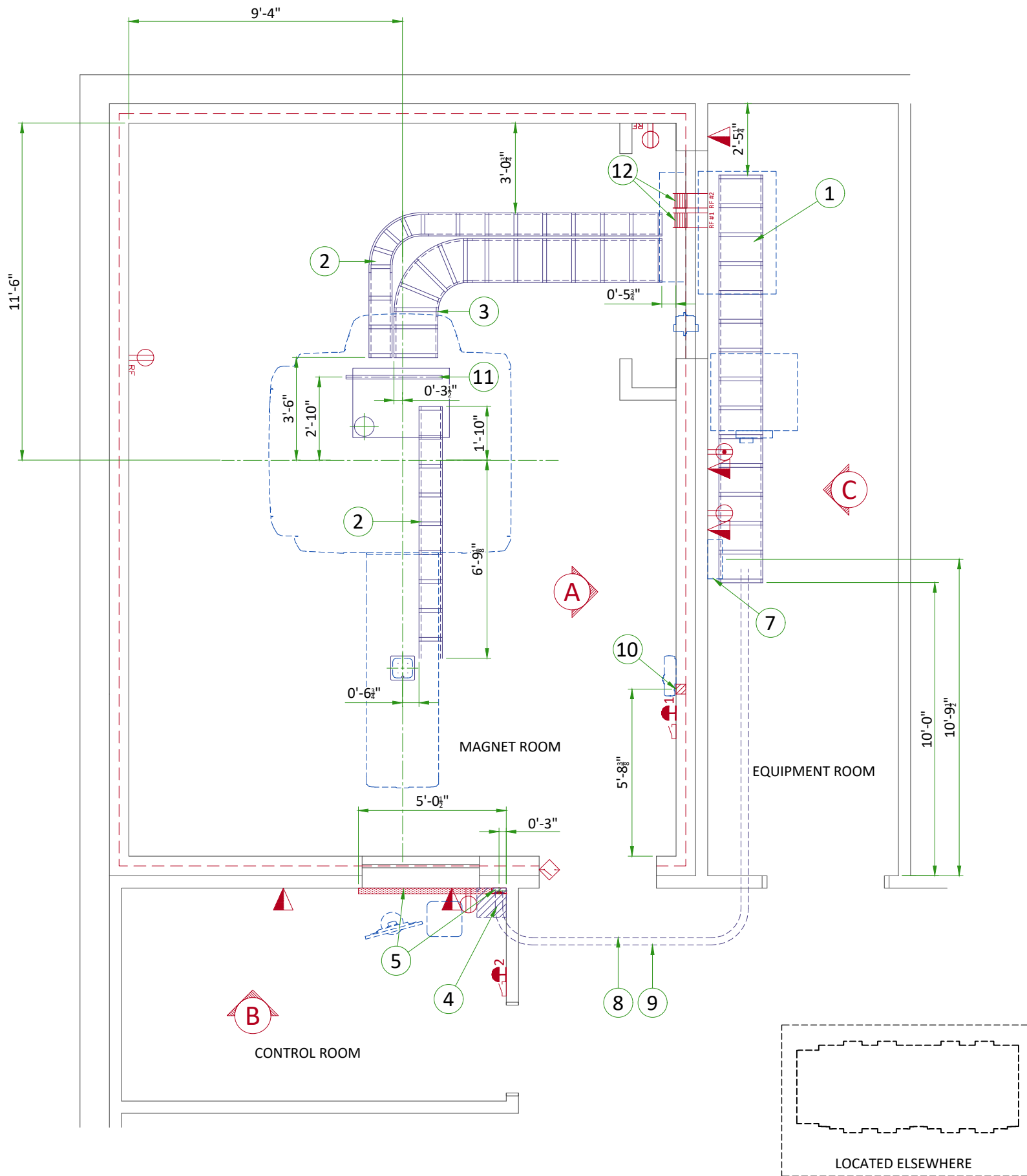


ITEM	Electrical Layout Item List
1	Cable ladder 18" x 6" [450 x 150]
2	Non-ferrous cable ladder 8" x 6" [200 x 150]
3	Non-ferrous cable ladder for gradient cables 18" x 6" [450 x 150]
4	Box above ceiling size per local code
5	Surface wall duct with minimum 2 dividers 6' x 3 1/2" [150 x 100]
6	Box above ceiling 4" x 4" x 4" [100 x 100 x 100] (IRC)
7	Facility disconnect panel (FD). Refer to Power Requirements detail.
8	Conduit above RF screen 2" [50]
9	Conduit above RF screen 3" [75]
10	Box in wall 4" x 4" x 4" [100 x 100 x 100] (MRU)
11	Non-ferrous unistrut cable support
12	RF Filters - grounded to RF shield at Common Ground Stud

ITEM	QTY	Electrical Outlet Legend
Customer/contractor supplied and installed items unless otherwise specified. Height above floor determined by local codes unless otherwise specified.		
		System emergency off (EO1-2), (recommended height 1.2m [48"] above floor)
		Door interlock switch
		Emergency exhaust fan switch 1.2m [48"] height recommended)
		Duplex hospital grade, dedicated wall outlet 120-v, single phase power
		Network outlet
		Dedicated telephone lines/network connection
		Duplex hospital grade, dedicated outlet 120-v emergency, single phase power, 15a
		Duplex hospital grade, dedicated outlet 120-v, single phase outlet routed through RF filter


Additional Conduit Runs (Contractor Supplied and Installed)					
	From (Bubble # / Item)	To (Bubble # / Item)	Qty	Size	
				In.	mm
7	Facility Disconnect (FD)	Facility Power	1	As req'd	As req'd
7	Facility Disconnect (FD)	Power Distribution Unit (inside ISC)	1	As req'd	As req'd
		System emergency off 2	1	1/2	16
	System emergency off 2	Penetration Panel	1	1/2	16
	Door switch	Integrated Systems Cabinet	1	3/4	20
	System emergency off 1	Penetration Panel	1	3/4	20
6	Magnet Rundown Unit	Magnet	1	1	25
		Integrated Systems Cabinet	1	As req'd	As req'd
	RF filter #1	120-V 1Ø Power	1	As req'd	As req'd
	Room Light	RF filter #2	1	As req'd	As req'd
	RF filter #2	Facility emergency power	1	As req'd	As req'd
	Dimplex Chiller	Remote Graphic Display	1	3/4	20
		Facility Power	1	As req'd	As req'd

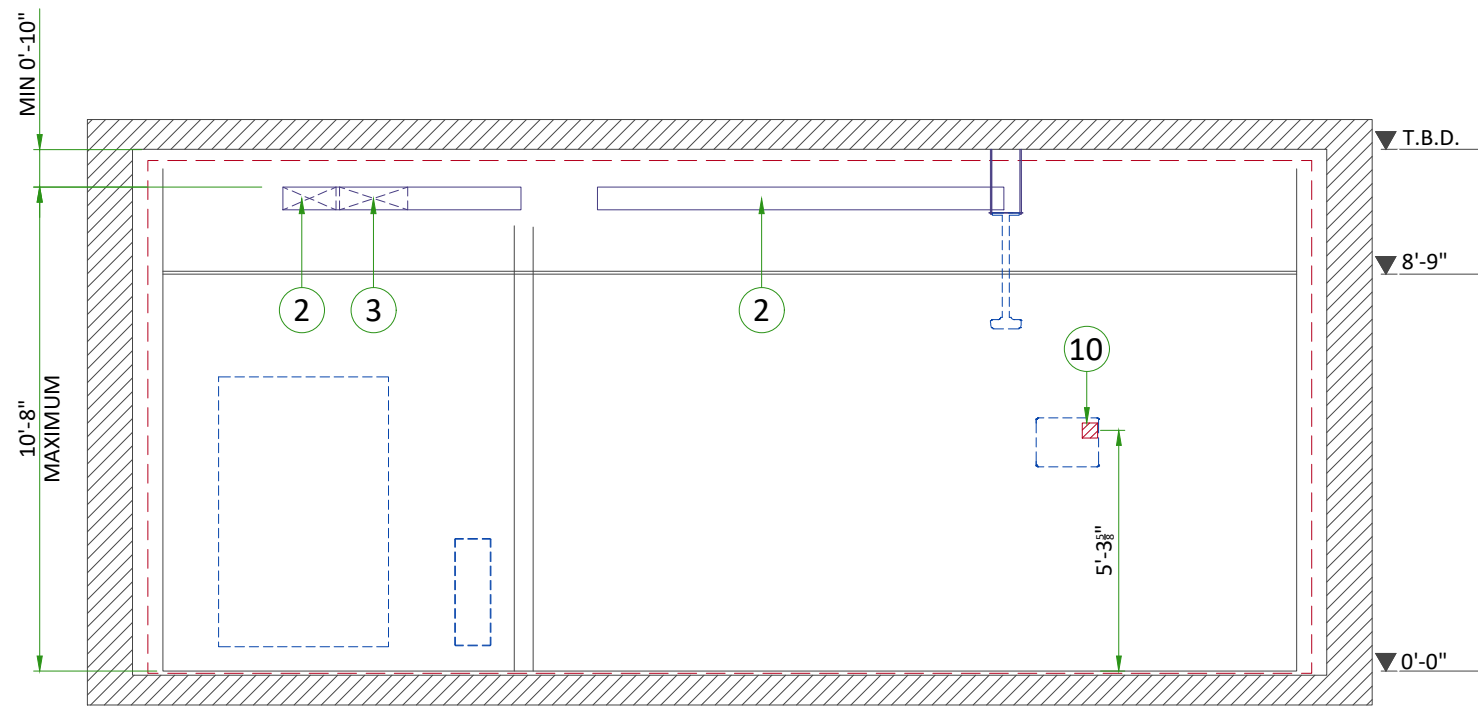

A

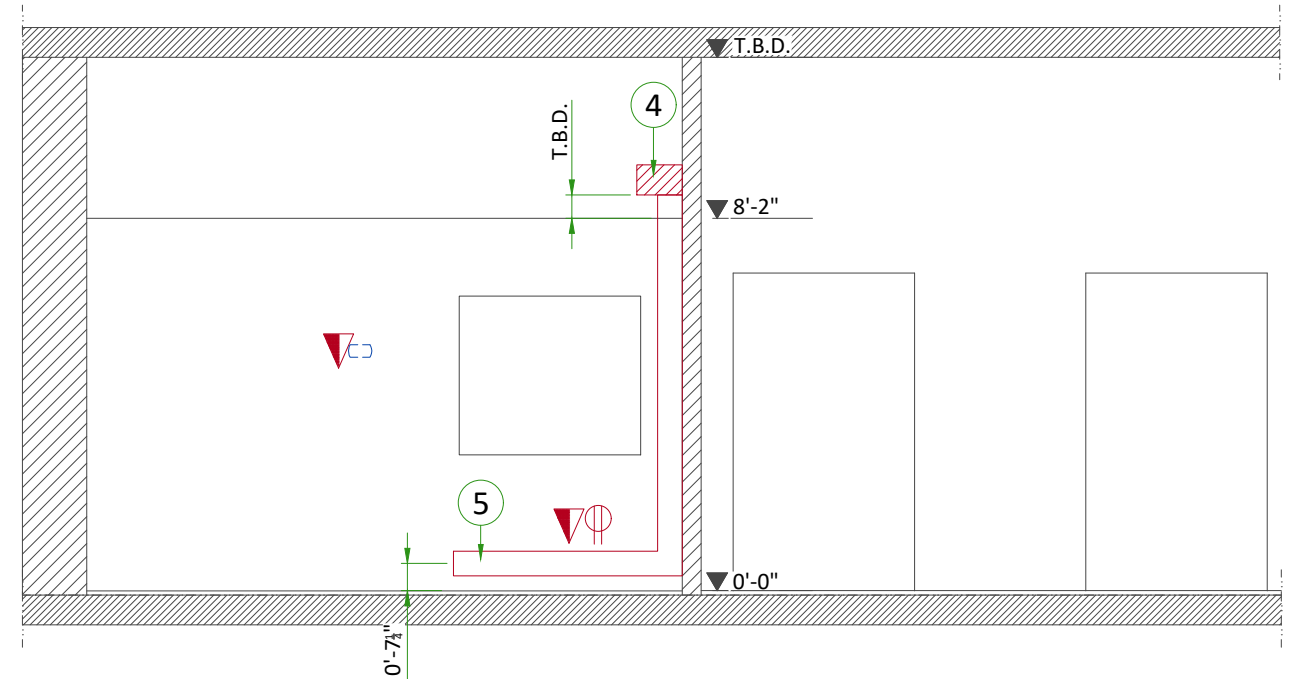

B

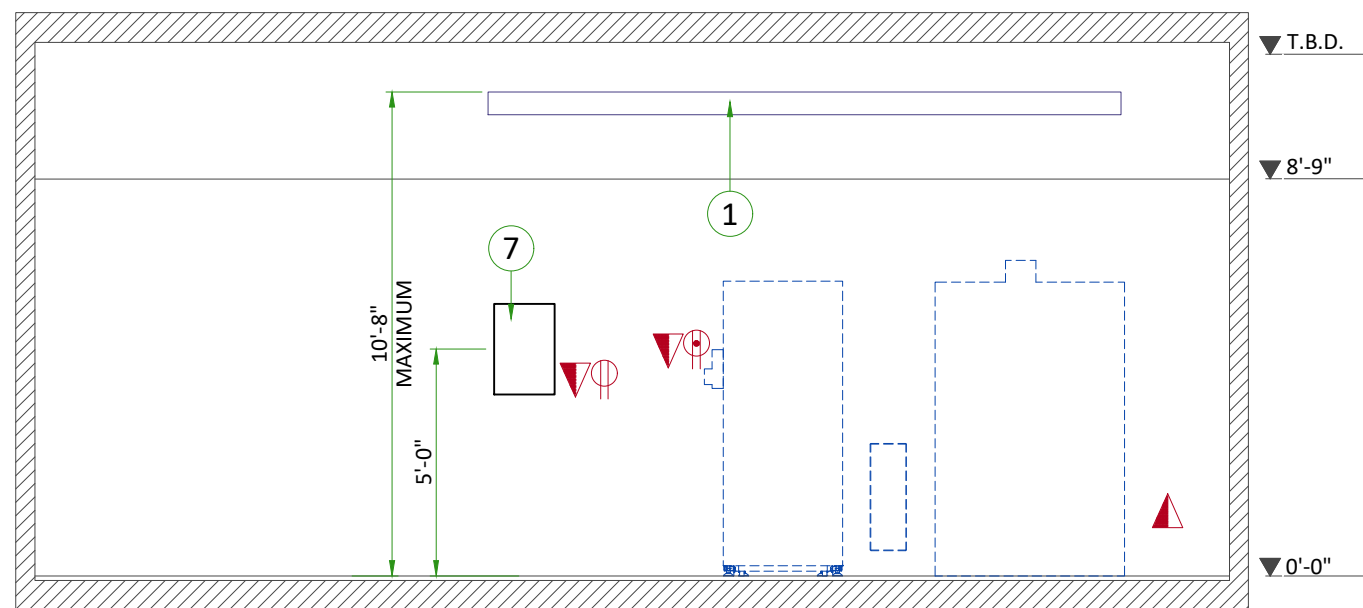


C

D


CABLE TRAYS IN EQUIPMENT ROOM

CABLE TRAY DETAIL

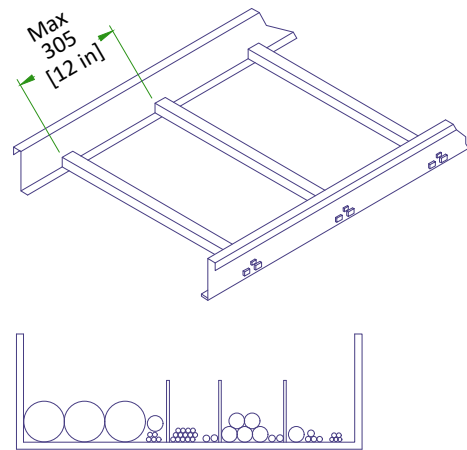


Minimum Cable Tray Width			
From	To	Width	For
ISC	ICC	229 [9 in]	Electrical, Water
ISC	CAB	76 [3 in]	Electrical
ICC	PP	300 [12 in]	Electrical, Water, Helium lines

NOT TO SCALE

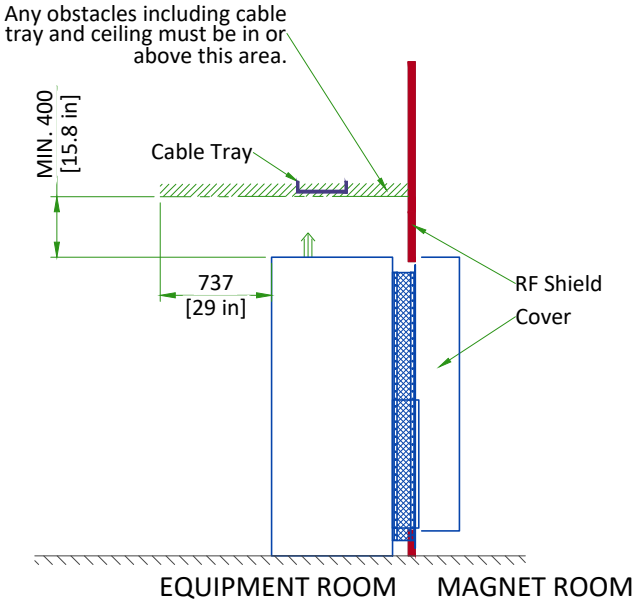


CABLE TRAYS REQUIREMENTS IN MAGNET ROOM

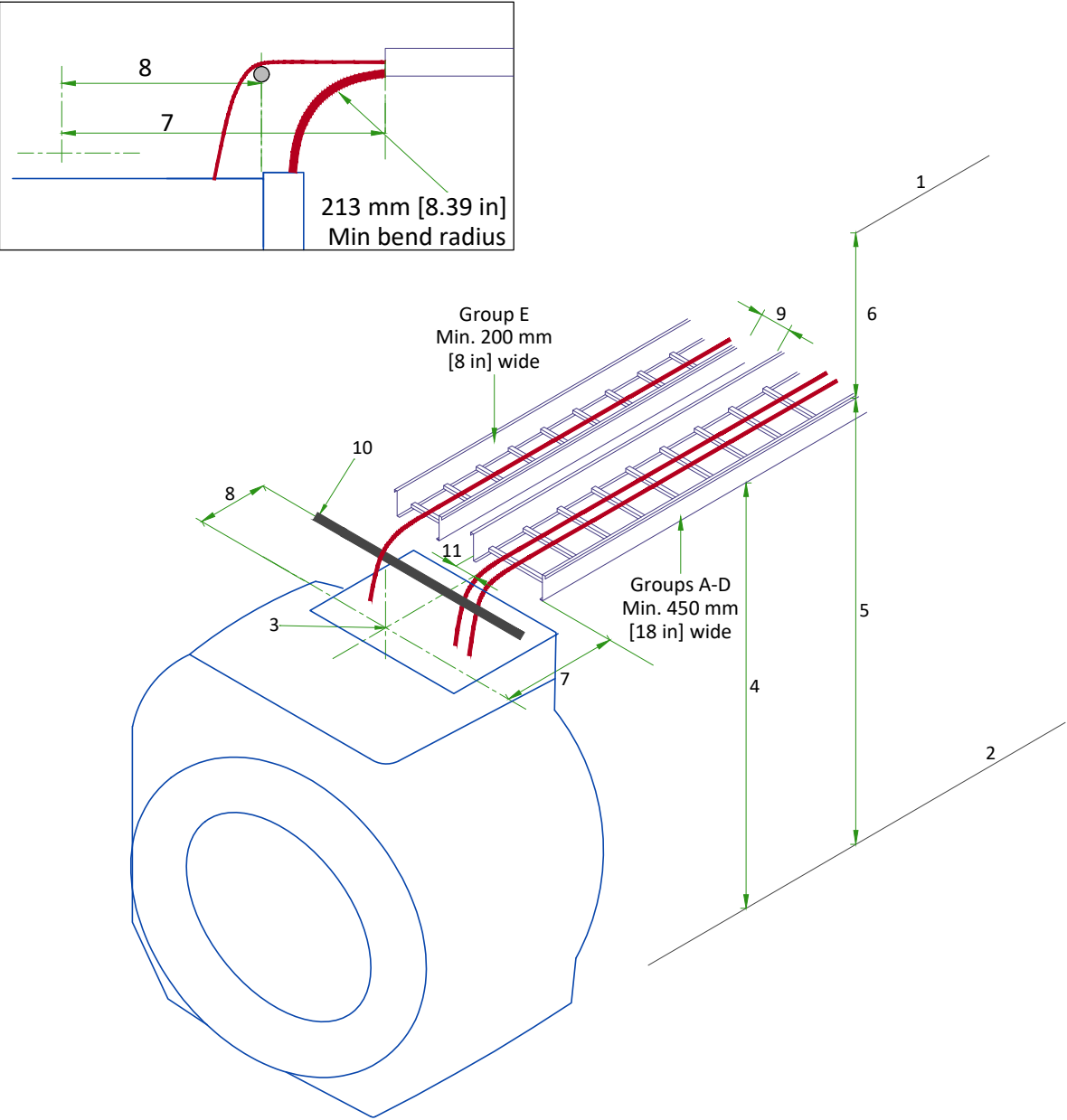

CABLE TRAY REQUIREMENTS (SIDE-BY-SIDE)

- 1 - Ceiling
- 2 - Finished Floor
- 3 - Magnet isocenter
- 4 - Minimum cable tray height for Groups A,B,C,D required at back of Magnet: 2449 mm [96.4 in].
- 5 - Maximum height from floor to top of tray (anywhere in Magnet room): 3251 mm [128 in].
- 6 - 152 mm [6 in] if cables are accessible from both sides of the cable support or accessible from beneath the cable support. 254 mm [10 in] if the cables are accessible from only one side of the cable support and also not accessible from beneath the cable support.
- 7 - Tray end to isocenter: 1067 mm [42 in].
- 8 - Other cable termination to isocenter: 864 ±12 mm [34 ±0.5 in].
- 9 - Minimum distance between trays: 12 mm [0.5 in].
- 10 - Non-ferrous cable support
- 11 - The center of the gradient cable group is 89 mm [3.5 in] from the inside edge of the tray, in line with the magnet center.

FACILITY SUPPLIED WIRING

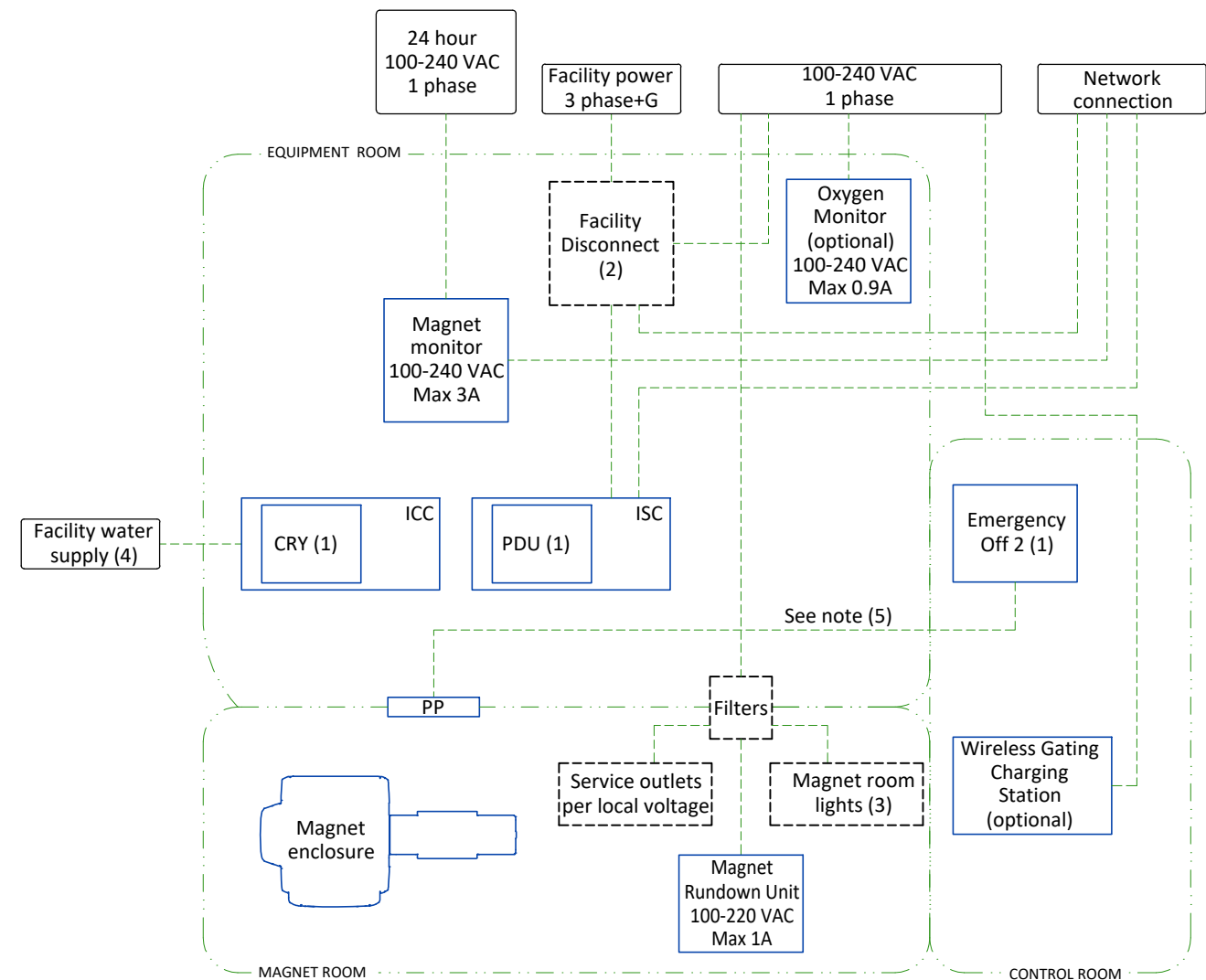

This diagram displays minimum power requirements for GEHC equipment and should be used as a guide to determine appropriate wire sizes per local regulatory requirements.

- CRY Cryocooler Compressor (Inside ICC)
ICC Integrated Cooling Cabinet
ISC Integrated System Cabinet. Shown in off the wall configuration, may also be mounted in wall with PP.
PDU Power Distribution Unit (inside ISC)
PP Penetration Panel

- Notes :**
- (1) Refer to Interconnect detail for more information
 - (2) Customer is required to install a facility disconnect (FD) that is accessible to service personnel in the Equipment Room. The FD shall have provisions for industry standard Lock Out Tag Out (LOTO).
 - (3) Refer to Lighting Requirements detail
 - (4) This group contains water lines which shall be routed separate from electrical lines (I.E. power and signal)
 - (5) A cable is supplied by GEHC but may be extended if needed.



POWER REQUIREMENTS

SPECIFICATIONS OF MAIN POWER INPUT

POWER SUPPLY	380/400/415/480V +/-10%, 3 PHASE + GND
FREQUENCIES	50/60Hz ± 3Hz
ISC PDU CONTINUOUS POWER	55 kVA
TOTAL SYSTEM 50 MILLISECOND POWER	80 kVA
TOTAL SYSTEM CONTINUOUS POWER	64 kVA

- **Lock-out/Tag-out:** Customer is required to install a facility disconnect (FD) that is accessible to service personnel in the Equipment Room. The FD shall have provisions for industry standard Lock Out Tag Out (LOTO).
- The facility must supply and install wiring for system power to the terminals on the top of the Integrated System Cabinet (ISC).
- Governing electrical codes may require a neutral wire. If present, neutral must be terminated in the ISC.
- Power input must be separated from any others which may generate transients (elevators, air conditioning, radiology rooms equipped with high speed film changers...).
- Recommended voltage total harmonic distortion less than 2.5%. Voltage phase imbalance must not exceed 2%.

cPDU CIRCUIT BREAKER CURRENT RATING	
Nominal input voltage (V)	Circuit breaker current (A)
380	125
400	120
415	115
480	100

ISC PDU TERMINALS Range of standard conductors accepted		
Connection	mm²	AWG
Phase	16-95	6-3/0
Ground	16-95	6-3/0

SPECIFICATIONS OF OPTIONAL BACK-UP POWER SUPPLY

MAGNET MONITOR REQUIRES A 110/220 VAC, 50/60 HZ, 3.0 A FACILITY SUPPLIED OUTLET. POWER AT THE OUTLET MUST BE CONTINUOUSLY AVAILABLE.


FOR CRYOCOOLER COMPRESSOR	
POWER INPUT	380/400/415/480V, THREE-PHASE
POWER DEMAND	MIN 9kVA/RECOMMENDED 12 kVA
FREQUENCY	50/60Hz ± 3Hz
Power to Cryocooler Compressor must be removed when emergency off circuit is actuated.	

- A cryocooler power bypass kit is available to provide power to the cryocooler compressor before the ISC is installed.

CABLES

- Power and cable installation must comply with the distribution diagram.
- All associated transformers and cables must be correctly sized for system power requirements.
- All cables must be stranded copper and cable color codes must comply with standards for electrical installation.
- Facility electrical cables must not be routed in the same cable supports as the MR system cables.
- Each conductor will be identified and isolated (screw connector).


POWER DISTRIBUTION AND EMERGENCY OFF CONNECTION

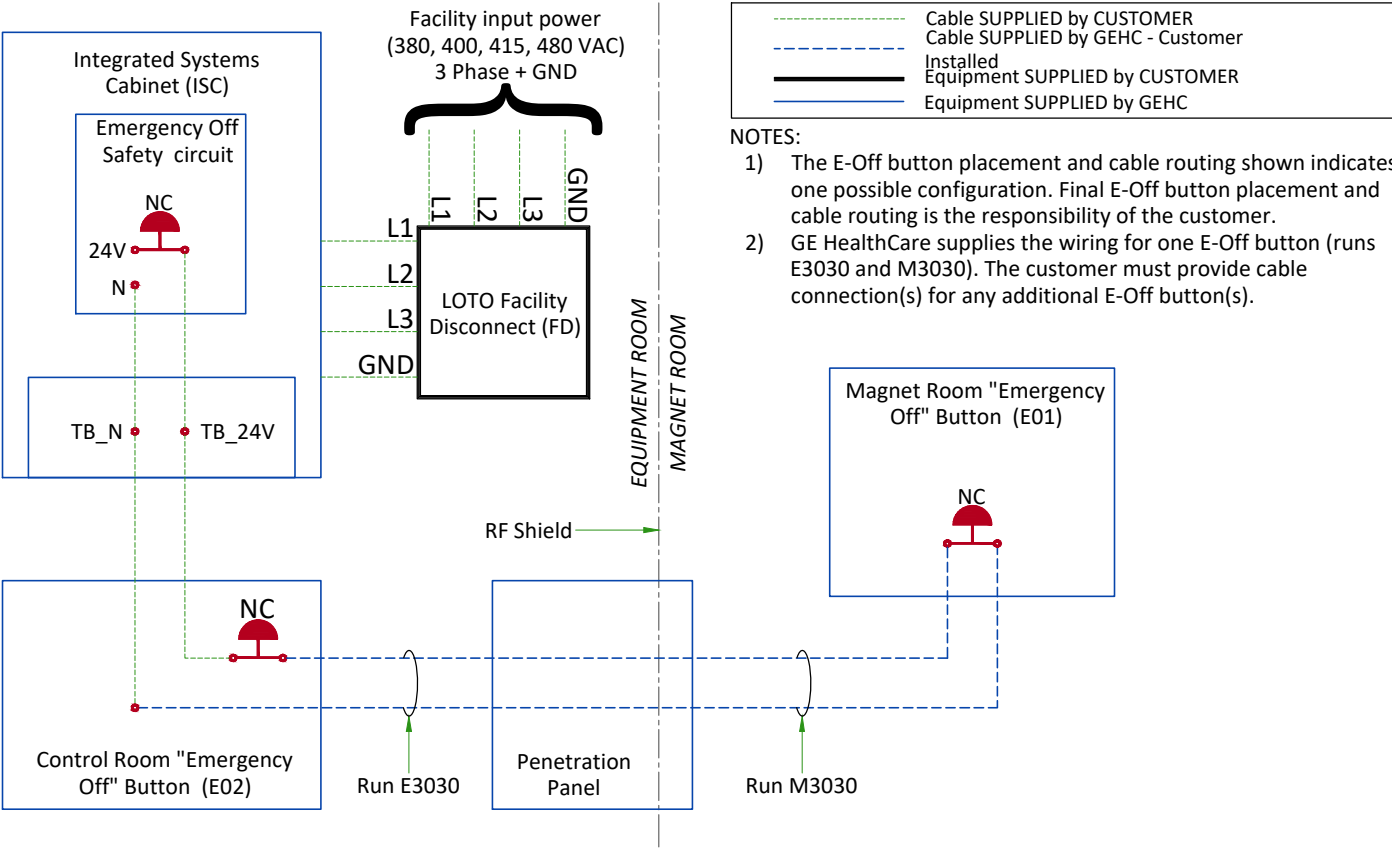


TYPICAL MAGNET ROOM GROUNDING

GROUNDING REQUIREMENTS

- All power lines into the RF shielded room require an RF filter.
- All electrical devices (for example, RF filters, outlets, light fixtures, and so on) must have a ground wire from device power source and be grounded to the RF Shield at the RF Common Ground Stud.
- Resistance between any two grounded devices must not exceed 0.1 ohm to ensure equal potential ground system within the Magnet Room.
- Do not ground non-MR equipment to the MR ground system.
- All ground wires for the MR system, including wires between the MR system and the RF Common Ground stud, are provided by GE HealthCare.
- The common ground stud must be installed within 1.5 m [5 ft] of all penetration point(s) of the GE HealthCare equipment, into the RF shield between the Equipment Room and Magnet Room.
- The stud must be expanded with a ground bus on each side (Equipment Room and Magnet Room) with at least 6 additional ground studs or threaded holes on each side. Additional studs/holes must be M6 [¼ in] and include appropriate fastening hardware (lock washers, nuts, bolts).
- For additional information refer to RF Shielded Room manual 5850260-1EN

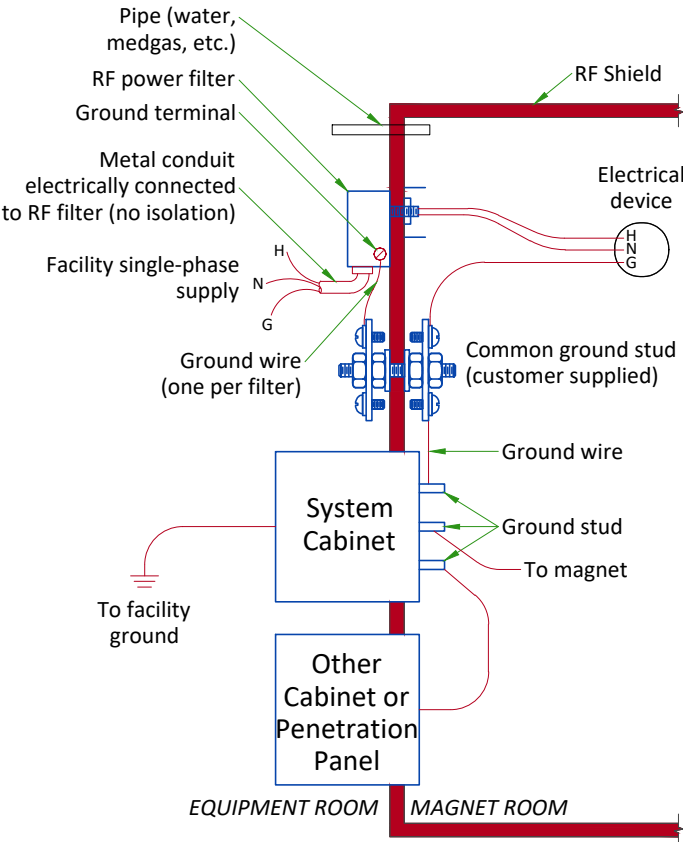

INTERCONNECTIONS

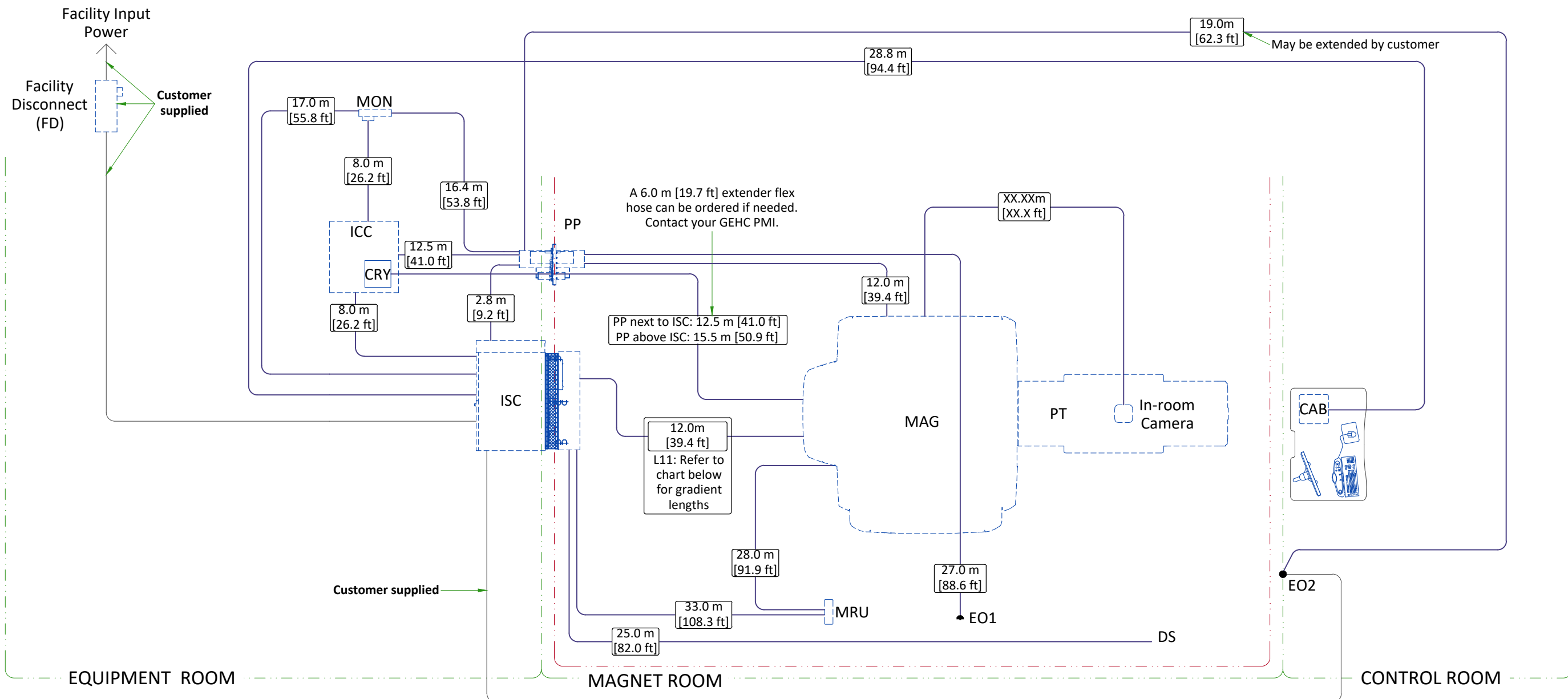

GRADIENT CABLE LENGTH OPTIONS		
Length Identifier	Available lengths m (ft)	Proposed
L11 (Gradient Cable)	4.6 (15.1)	-
	6.6 (21.6)	-
	8.6 (28.2)	-
	10.6 (34.7)	-
	12.6 (41.3)	-
L11 (RF Cables)	9.0 (29.5)	-
	13.0 (42.7)	-

GENERAL NOTE: PMI must validate proposed selectables and send confirmation to OTR. Refer to MyProjects if Proposed field is blank.

GOLDSEAL/SILVER PREFERRED NOTE: Cable lengths listed may differ from what is shipped with the system. Contact the Goldseal group for actual lengths to be delivered.

ROOM MOVE NOTE: Cable lengths listed may differ from what is included with reinstalled system. Contact the local field engineer for actual lengths to be delivered.

CABLES ROUTING FOR OPTIONS					
Option	From	To	Cable Length		
			Note	m	ft
fMRI	Magnet Isocenter	CAB		31.5	103
	ISC, IO Panel	CAB		25	82
Magnetic Resonance Elastography (MRE)	MRE	Magnet Isocenter	Nominal	7.31	24
			Maximum	10.06	33
	MRE	ISC cabinet		15.24	50
	MRE	Ethernet Hub in ISC		15.24	50
	MRE	Customer Supplied Outlet	60 Hz	6.09	20
			50 Hz	7.62	25